**CerTest** Rapid Test

**CerTest**  
BIOTEC

**CerTest**

***C. difficile* GDH+Toxin A+B + Lactoferrin**

ONE STEP *C. difficile* antigen GDH, Toxin A,  
Toxin B and Lactoferrin COMBO CARD TEST

CE 

The test should be used by professional trained in IVD devices

These instructions for use apply to the following references:

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**LX862001VF; LX862001VFC; LX862001VFCC**

Initial part of the Instructions for use is common for all the references/

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Specific parts for each of the references are described in the corresponding annexes (see tables below)/

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Ref. LX862001VFCC

C- (x2) C+ (x4) (x6) (x4) (x2)

**2a** Procedure for GDH, Toxin A and Toxin B.

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---

## 2. Intended use

**CerTest C. difficile GDH+Toxin A+B+Lactoferrin** one step combo card test is a coloured chromatographic immunoassay for the simultaneous qualitative detection of *Clostridium difficile* Glutamate Dehydrogenase (GDH), Toxin A and Toxin B in stool samples and for the semi-quantitative detection of human lactoferrin (hLf) in stool samples.

**CerTest C. difficile GDH+Toxin A+B+Lactoferrin** combo card test offers a simple and highly sensitive screening assay to make a presumptive diagnosis of *Clostridium difficile* infection and intestinal inflammation presence.

## 3. Summary and Explanation

*Clostridium difficile* (*C. difficile*), a Gram-positive spore bearing anaerobic bacterium is the major aetiological agent of diarrhoea and colitis associated with antibiotics (1).

*C. difficile* is the most common cause of health care-associated diarrhoea in developed countries and is a major source of nosocomial morbidity and mortality worldwide. Although traditionally considered an infection affecting patients in health care facilities, in the last 10-20 years *C. difficile* infection is increasingly identified in the community among people with no recent hospitalizations and without prior antibiotic exposure (2-4).

Disease due to *C. difficile* develops when the bacteria is allowed to proliferate in the colon, most commonly after antibiotic use has resulted in sufficient disturbance of the intestinal microflora. Overall, the use of antibiotics is associated with a three-fold increased risk of community-associated *C. difficile* infection (CA-CDI). However, the risk of CA-CDI is largely dependent on the class of antibiotic use (1,3).

*C. difficile* can release two high-molecular-weight toxins, Toxin A and Toxin B, which are responsible for the clinical manifestations, which range from mild, self-limited watery diarrhoea to fulminant pseudomembranous colitis, toxic megacolon, and death. *Clostridium difficile* Glutamate Dehydrogenase (GDH) is an enzyme produced in large quantities by all toxigenic and non-toxigenic strains, making it an excellent marker for the microorganism (2,5).

The use of a direct faecal GDH screen, together with a faecal Toxin AB test, could improve the diagnosis of *Clostridium difficile* infection. Patients with a positive faecal GDH but negative toxin can be monitored closely while awaiting the result of a toxigenic culture (TC), that is used as the gold standard technique to determine *Clostridium difficile* infection (5).

Preventing *C. difficile* infection offers a potentially significant improvement in patient's outcomes, as well as a reduction in hospital costs and resource expenditures. Careful use of antibiotics remains a prominent public health strategy to reduce *C. difficile* infection cases (1,3).

Lactoferrin is a glycoprotein component of neutrophil secondary granules, a primary component of the acute inflammatory response and is released from faecal leukocytes. This protein is resistant to proteolysis in the

faeces and may serve as a marker of inflammation in the intestine. The major cause of faecal neutrophils in patients with chronic diarrhoea is chronic inflammatory bowel disease of the colon (i.e., Crohn's Disease and ulcerative colitis). Lactoferrin has been also studied as a predictor of infection with invasive enteropathogens in children with diarrhoea. Bacterial inflammatory diarrhoea may be caused by *Shigella*, *Salmonella*, *Campylobacter* and *Clostridium difficile* (6,7).

## 4. Test principle

**CerTest C. difficile GDH+Toxin A+B+Lactoferrin** is based on the principle of a qualitative immunochromatographic assay for the qualitative detection of *Clostridium difficile* Glutamate Dehydrogenase (GDH), Toxin A, Toxin B and Lactoferrin in human stool samples from patients suspected of *Clostridium difficile* infection and intestinal inflammation presence.

**Strip A** consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against GDH and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-GDH) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming two coloured conjugate complexes.

**Strip B** consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against Toxin A and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-Toxin A) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming two coloured conjugate complexes.

**Strip C** consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against Toxin B and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-Toxin B) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming two coloured conjugate complexes.

**Strip D** consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against human lactoferrin and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-human lactoferrin) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming two coloured conjugate complexes.

If the sample is GDH positive, the antigen of the diluted sample reacts with the red-coloured conjugates complex (anti-GDH monoclonal antibodies-red polystyrene microspheres) in the strip A, if the sample is Toxin A positive, the antigens of the diluted sample react with the red-coloured conjugates complex (anti-Toxin A monoclonal antibodies-red polystyrene microspheres) in the strip B, if the sample is Toxin B positive, the

antigens of the diluted sample react with the red-coloured conjugated complex (anti-Toxin B monoclonal antibodies-red polystyrene microspheres) in strip C and if the sample is human lactoferrin positive, the antigens of the diluted sample react with the red-coloured conjugated complex (anti-human lactoferrin monoclonal antibodies-red polystyrene microspheres) in strip D, which were previously pre-dried on the absorbent pad. The mixture then moves upward on the membrane by capillary action. As the sample flows through the test membrane, the binding conjugate complexes migrate. The anti-GDH antibodies present on the membrane of strip A (test line), the anti-Toxin A antibodies present on the membrane of strip B (test line), the anti-Toxin B antibodies present on the membrane of strip C (test line) and the anti-human lactoferrin antibodies present on the membrane of strip D (test line) capture the coloured conjugate and the red lines will be visible in the three strips. These bands are used to interpret the result.

If the sample is negative, there is no GDH, Toxin A, Toxin B and human lactoferrin presence and yet, the antigens may be present in a concentration lower than the detection limit value, for which the reaction will not take place with any red-coloured conjugate complex. The anti-GDH, the anti-Toxin A, the anti-Toxin B and the anti-human lactoferrin antibodies present on the membranes (test lines) will not capture the antigen-red-coloured conjugate complex (not formed), for which the red lines will not appear.

Whether the sample is positive or not, in the three strips, the mixture continues to move across the membranes to the immobilized specific antibodies placed in the control lines. The anti-specific protein antibodies present on three membranes will capture control green-conjugate complex and three control lines will always appear. The presence of these green lines serve as: 1) verification that sufficient volume is added, 2) that proper flow is obtained and 3) an internal control for the reagents.

## 5. Storage and Stability

The kits should be stored at 2-30°C, in these conditions can be used until the expiration date which is stated on the kit label. All the kit components are for only one use and should remain in its primary packaging until use. Do not freeze.

## 6. Precautions for users

- **For professional *in vitro* diagnostic use only.**
- Read the instructions for use carefully before use the kit.
- Do not use after expiration date.
- Do not use the kit if the label that seals the outer box is broken.
- Do not use product if the protective external box or the protective aluminium pouches are opened or damaged upon arrival.
- Do not use the test (and positive/negative controls if the final reference contains them) if desiccant material is not present or broken inside the aluminium pouch.

- Components provided in the kit are approved for use in the **CerTest C.difficile GDH+Toxin A+B+Lactoferrin**. Do not mix or use with components from other product kit or lots. Do not use with other commercial kit components (from another supplier).
- All the components of the kits are for single use. Do not reuse.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves or other personal protective equipment such as goggles and mask that will be considered necessary. Do not eat, drink or smoke in the working area.
- All the specimens should be considered potentially hazardous, as well as all the materials that have been exposed to the samples and they must be handled according to the local or national safety regulations. Must be handled in the same manner as an infectious agent. Use proper infection control practices. These practices should include, but are not limited to, personal protective equipment (PPE), such as laboratory coat, surgical or appropriate mask, or face shield, disposable gloves, and eye protection. Take necessary precautions during the collection, transport, storage, handling, and disposal of the samples. Each sample must be correctly and unequivocally identified in order to guarantee the correct traceability of the samples.
- A new test must be used for each sample to avoid contaminations errors.
- Clean up spills thoroughly using an appropriate disinfectant.
- Reagents contain preservatives (<0.1% sodium azide). Avoid any contact with skin or mucous membrane. In accordance with Regulation (EC) No 1907/2006 (REACH), **CerTest C.difficile GDH+Toxin A+B+Lactoferrin** do not contain substances and/or mixtures which meet the hazard classification criteria available in Regulation (EC) No 1272/2008 (CLP) or which are in concentrations higher than the value established in the mentioned regulation for their declaration. Material Safety Data Sheet are not included with this device, nevertheless it could be consulted upon requested to CerTest Biotec.
- The certificate of analysis is not included with the device, however, it could be requested to CerTest Biotec in case of need.
- The presence of yellow lines in the results window (control line zone and test line zone), before using the test, is completely normal and does not imply failure of the test functionality.
- Visual interpretation of the results by coloured lines, interpretation of the results must be done by professional user without problems in colour interpretation.
- The tests (vials, devices, positive/negative control if any) should be discarded in a proper biohazard container after testing. These containers should be discarded in accordance with local or national laws or regulations.

## 7. Specimen collection and preparation

Samples should be collected in a clean container and processed as soon as possible after collection. Follow proper infection control practices.

Samples can be preserved at room temperature if they are going to be analysed into the same day after taking them. The recommendation is processed the sample immediately after collection.

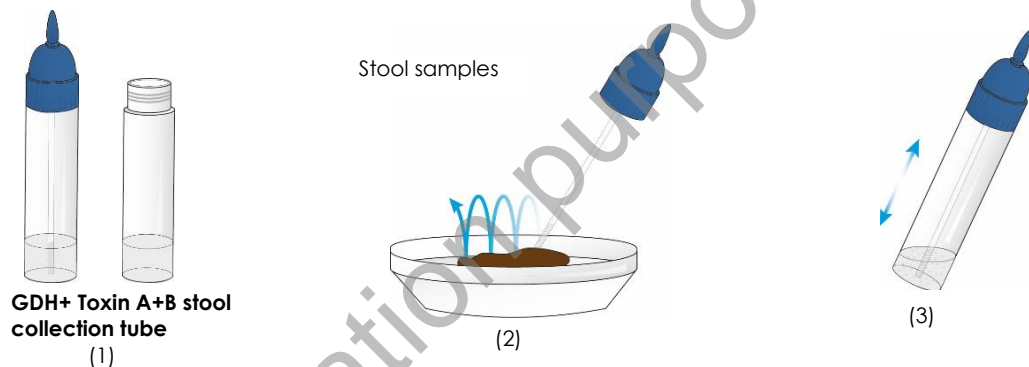
If this is not possible, stool samples could be stored in the refrigerator (2-8°C) for a maximum of 24 hours. For longer storage, maximum 1 year, the specimen must be kept frozen (-20°C). In this case, the sample will be totally thawed and brought to room temperature before testing. Ensure only the amount needed is thawed because of freezing and defrosting cycles are not recommended. Homogenise stool samples as thoroughly as possible prior to preparation.

**Please take note that there are two different stool collection tubes:**

**a. GDH+Toxin A+B stool collection tubes**

**Specimen preparation (see illustration):**

1. Take out the cap of the **GDH+Toxin A+B stool collection tube** (1) and use the stick to pick up sufficient sample quantity. Then, introduce the stick once into 4 different parts of the stool sample (2), to collect faecal sample (approx. 50 mg) and add it to the **GDH+Toxin A+B stool collection tube**. For liquid samples, add approx. 125 µL in the **GDH+Toxin A+B stool collection tube** using a micropipette.
2. Close the **GDH+Toxin A+B stool collection tube**. Shake the tube in order to assure good sample dispersion (3).


Continue immediately with the test procedure. See Annexes. Each sample vial (containing the sample) and cassette with which will be tested that sample should be perfectly recognizable including patient's identification.

**b. Lactoferrin stool collection tubes**

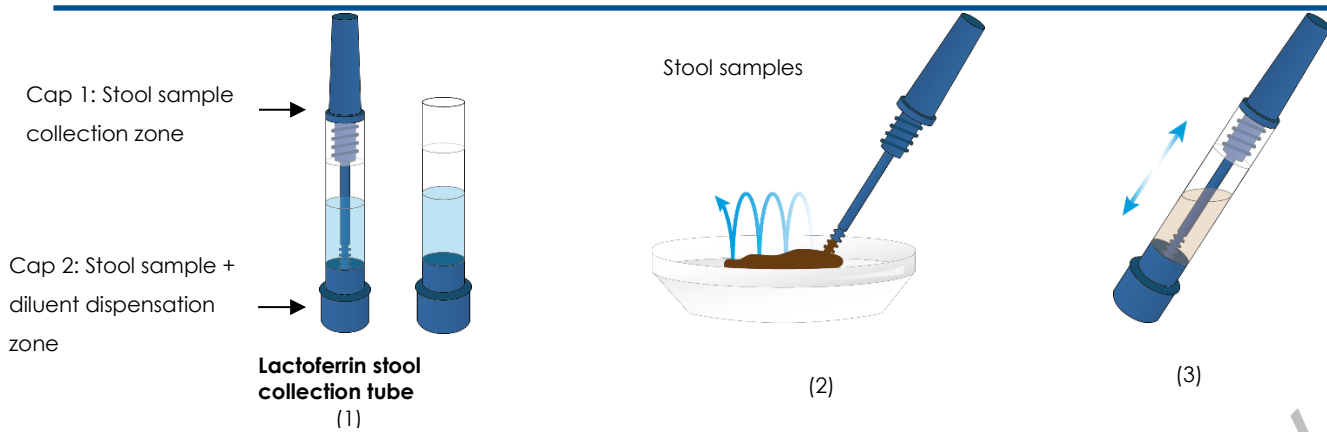**Specimen preparation (see illustration):**

1. Take out the cap 1 of the **Lactoferrin stool collection tube** (1) and use the stick to pick up sufficient sample quantity. Then, introduce the stick once into 4 different parts of the stool sample (2), to collect faecal sample (approx. 20 mg) and add it to the **Lactoferrin stool collection tube**. For liquid samples, add approx. 20 µL in the **Lactoferrin stool collection tube** using a micropipette.
2. Close the **Lactoferrin stool collection tube**. Shake the tube in order to assure good sample dispersion (3).

Continue immediately with the test procedure. See Annexes. Each sample vial (containing the sample) and cassette with which will be tested that sample should be perfectly recognizable including patient's identification.

## 8. Limitations of the test

- 1) The test must be carried out within 2 hours after opening the sealed bag.
- 2) An excess of sample could cause wrong results (brown bands appear). Dilute the sample with the diluent and repeat the test.
- 3) The intensity of test line may vary from very strong at high antigens concentration to faint when the antigens concentration is close to the detection limit value of the test.
- 4) **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** should be used only with samples from human faeces. The use of other samples has not been established. The quality of the test depends on the quality of the sample; proper faecal specimens must be obtained.
- 5) Positive results determine the presence of GDH, Toxin A, Toxin B of *Clostridium difficile* and human lactoferrin in faecal samples. A positive result should be followed up with additional laboratory techniques (toxigenic culture) to determine the strain of *Clostridium difficile*. A confirmed infection should only be made by a physician after all clinical and laboratory findings have been evaluated and must be based in the correlation of the results with further clinical observations.
- 6) A negative result is not meaningful because of it is possible GDH, Toxin A, Toxin B of *Clostridium difficile* and/or human lactoferrin concentration in the stool sample is lower than the detection limit value or cut-off value. If the symptoms or situation persist, a *Clostridium difficile* determination should be carried out, on a sample from an enrichment culture.
- 7) Lactoferrin is a component of breast milk; the test will be positive in breast fed children and should not be used to evaluate neonates receiving breast milk.
- 8) Stool from patients with active inflammatory bowel diseases that usually involve significant neutrophilic inflammation of the intestine, such as Crohn's disease and ulcerative colitis, would be positive for faecal lactoferrin.

- 9) Bloody stool samples can contain components that may cause non-specific reactions in the test. Every bloody stool samples whose result is positive should be followed up with other techniques of diagnosis to confirm the result.
- 10) Mucous stool samples may cause non-specific reactions in the test. Every mucous stool sample whose result is positive should be followed up with other techniques of diagnosis to confirm the result.

## 9. Expected values

*Clostridium difficile* is associated with 95-100% cases of pseudomembranous colitis, 60-75% cases of antibiotic-associated colitis and 35% of cases of antibiotic-associated diarrhoea cases. In addition, nearly 223.900 people in the United States required hospital care for *C. difficile* and at least 12.800 people died in 2017. Although U.S. outpatient antibiotic prescription rate decreased from 2011 to 2016, at least 30% of outpatient antibiotic prescription are estimated to be unnecessary, which highlights the need to improve outpatient prescribing (4,5,8).

While the development of new treatments is one of these key actions, such investments must be coupled with dedicated efforts toward preventing infections in the first place, slowing the development of resistance through better antibiotic use, and stopping the spread of resistance (8).

Some studies established equal or higher 10 µg hLf/g faeces as cut-off value to allow to detect adult patients with gastrointestinal inflammatory pathology that will require to diagnosis additional diagnostic invasive procedures(6).

## 10. Performance characteristics

### A. Analytical sensitivity (detection limit)

Detection limit value (LoD) of **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** is 0.39 ng/mL of *Clostridium difficile* antigen GDH, 2 ng/mL of *Clostridium difficile* Toxin A, 3.12 ng/mL of *Clostridium difficile* Toxin B and 100 ng/mL (10 µg hLf/ g faeces) of human lactoferrin.

There is not any external standard approved as a general and reference standard. The internal standard used for validation of this CerTest products are purified *Clostridium difficile* Glutamate dehydrogenase recombinant protein and purified *Clostridium difficile* Toxin A and Toxin B, and human lactoferrin.

### B. Clinical sensitivity and specificity

#### B.1 Clinical sensitivity and specificity -GDH+Toxin A+Toxin B

An external retrospective evaluation (Hospital Clinico Lozano Blesa, Zaragoza) with frozen stool samples was performed using **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** versus another immunochromatographic (IC) test. Discordant results between the IC tests were analyzed by qPCR. This is the evaluation criteria. The results were as follows:

**For Strip A: *Clostridium difficile* antigen GDH**

		Evaluation criteria (GDH)		
		+	-	Total
IC test: CerTest <i>C. difficile</i> GDH+Toxin A+B+Lactoferrin (GDH)				
	+	86	1	87
	-	3	160	163
	Total	89	161	250

Table 1 CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin (GDH) performance versus evaluation criteria.

CerTest <i>C. difficile</i> GDH+Toxin A+B+Lactoferrin (GDH) vs Evaluation criteria		
	Mean Value	95% confidence interval
Sensitivity	96.6%	90.5-99.3%
Specificity	99.4%	96.6-100.0%
PPV	98.9%	93.8-100.0%
NPV	98.2%	94.7-99.6%

Table 2 CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin (GDH) clinical results.**For Strip B: *Clostridium difficile* Toxin A**

		Evaluation criteria (Toxin A)		
		+	-	Total
IC test: CerTest <i>C. difficile</i> GDH+Toxin A+B+Lactoferrin (Toxin A)				
	+	51	0	51
	-	1	198	199
	Total	52	198	250

Table 3 CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin (Toxin A) performance versus evaluation criteria.

CerTest <i>C. difficile</i> GDH+Toxin A+B+Lactoferrin (Toxin A) vs Evaluation criteria		
	Mean Value	95% confidence interval
Sensitivity	98.1%	89.7-100.0%
Specificity	100.0%	98.2-100.0%
PPV	100.0%	93.0-100.0%
NPV	99.5%	97.2-100.0%

Table 4 CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin (Toxin A) clinical results.**For Strip C: *Clostridium difficile* Toxin B**

		Evaluation criteria (Toxin B)		
		+	-	Total
IC test: CerTest <i>C. difficile</i> GDH+Toxin A+B+Lactoferrin (Toxin B)				
	+	40	0	40
	-	4	206	210
	Total	44	206	250

Table 5 CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin (Toxin B) performance versus evaluation criteria.

CerTest <i>C. difficile</i> GDH+Toxin A+B+Lactoferrin (Toxin B) vs Evaluation criteria		
	Mean Value	95% confidence interval
Sensitivity	90.9%	78.3-97.5%
Specificity	100.0%	98.2-100.0%
PPV	100.0%	91.2-100.0%
NPV	98.1%	95.2-99.5%

Table 6 CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin (Toxin B) clinical results.

## B.2 Clinical sensitivity and specificity -Lactoferrin

An internal retrospective evaluation (Hospital Clinico Lozano Blesa, Zaragoza) with frozen stool samples was performed using **CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin** versus ELISA test. The ELISA technique has been considered the gold standard. This is the evaluation criteria. The results were as follows:

### For Strip D: Lactoferrin

#### Data for cut-off value 10µg hLf/g:

IC test: CerTest <i>C. difficile</i> GDH+Toxin A+B+Lactoferrin (Lactoferrin)	Evaluation criteria (Toxin B)		
		+	-
+	27	13	40
-	3	47	50
Total	30	60	90

Table 7 CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin (Lactoferrin) performance versus evaluation criteria.

CerTest <i>C. difficile</i> GDH+Toxin A+B+Lactoferrin (Lactoferrin) vs Evaluation criteria		
	Mean Value	95% confidence interval
Sensitivity	90.0%	73.5-97.9%
Specificity	78.3%	65.8-87.9%
PPV	67.5%	50.9-81.4%
NPV	94.0%	83.5-98.7%

Table 8 CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin (Lactoferrin) clinical results.

Specificity for a cut-off value of 2.5 µg hLf/g: 100.0% (mean value); 83.9-100.0% (95% confidence interval).

The most recent external retrospective evaluation performed to the test (with frozen stool samples) showed the following results:

- ***Clostridium difficile* antigen GDH:** sensitivity 96.6% (95% confidence interval: 90.5-99.3%) and specificity 99.4% (95% confidence interval: 96.6-100.0%).
- **Toxin A:** Sensitivity 98.1%, (95% confidence interval: 89.7-100.0%) and specificity 100.0% (95% confidence interval: 98.2-100.0%).

- **Toxin B:** Sensitivity 90.9%, (95% confidence interval: 78.3-97.5%) and specificity 100.0% (95% confidence interval: 98.2-100.0%).
- **Human Lactoferrin:** Sensitivity 90.0%, (95% confidence interval: 73.5-97.9%) and specificity 78.3% (95% confidence interval: 65.8-87.9%).

The results showed a high sensitivity and specificity to detect *Clostridium difficile* antigen GDH, Toxin A, Toxin B and/or human lactoferrin using **CerTest C. difficile GDH+Toxin A+B+Lactoferrin**.

#### **For Strip A: Clostridium difficile antigen GDH**

N°	Site	Sample type	Reference	Target
1	Hospital Clínico Lozano Blesa (Zaragoza, Spain) April-May 2021	Human stool samples	Code (17.1.1)	<i>Clostridium difficile</i> antigen GDH
2	Hospital Clínico Lozano Blesa (Zaragoza, Spain) April-June 2017	Human stool samples	Code (17.1.2)	<i>Clostridium difficile</i> antigen GDH
3	Louvain, (France) January- December 2016	Human stool samples	Code (17.1.3)	<i>Clostridium difficile</i> antigen GDH
4	ECCMID April 2016, Amsterdam	Human stool samples	Code (17.1.4)	<i>Clostridium difficile</i> antigen GDH

Table 9 Resume of Clinical evaluations. Site, sample type, target.

N°	TP	FP	FN	TN	N	Sens. (%) CI (%)	Spec. (%) CI (%)	PPV (%) CI (%)	NPV (%) CI (%)
1	86	1	3	160	250	<b>96.6</b> (90.5-99.3)	<b>99.4</b> (96.6-100.0)	<b>98.9</b> (93.8-100.0)	<b>98.2</b> (94.7-99.6)
2	51	0	0	75	126	<b>100.0</b> (93.0-100.0)	<b>100.0</b> (95.2-100.0)	<b>100.0</b> (93.0-100.0)	<b>100.0</b> (95.2-100.0)
3	130	8	8	332	478	<b>94.2</b> (88.9-97.5)	<b>97.6</b> (95.4-99.0)	<b>94.2</b> (88.9-97.5)	<b>97.6</b> (95.4-99.0)
4	27	8	6	117	158	<b>81.8</b> (64.5-93.0)	<b>93.6</b> (87.8-97.2)	<b>77.1</b> (59.9-89.6)	<b>95.1</b> (89.7-98.2)
<b>Total</b>	<b>294</b>	<b>17</b>	<b>17</b>	<b>684</b>	<b>1012</b>	<b>94.5</b> (91.4-96.8)	<b>97.6</b> (96.1-98.6)	<b>94.5</b> (91.4-96.8)	<b>97.6</b> (96.1-98.6)

Table 10 Clinical evaluations and global results. True positive (TP), false positive (FP), false negative (FN), true negative values (TN), and total samples (N). Sensitivity (Sens), specificity (Spec), Predictive Positive Values (PPV), Predictive Negative Values (NPV), 95% interval of confidence (CI) for CerTest C. difficile GDH+Toxin A+B+Lactoferrin (GDH).

#### **For Strip B: Clostridium difficile Toxin A**

Ref.	Location	Sample type	Reference	Target
1	Hospital Clínico Lozano Blesa (Zaragoza, Spain) April-June 2017	Human stool samples	Code (17.1.1)	<i>Clostridium difficile</i> Toxin A
2	Hospital Clínico Lozano Blesa (Zaragoza, Spain)	Human stool samples	Code (17.1.2)	<i>Clostridium difficile</i> Toxin A

	April-May 2021			
--	----------------	--	--	--

Table 11 Resume of Clinical evaluations. Location, sample type, target.

Ref.	TP	FP	FN	TN	N	Sens. (%) CI (%)	Spec. (%) CI (%)	PPV (%) CI (%)	NPV (%) CI (%)
1	28	0	1	97	126	<b>96.9</b> (92.2-99.9)	<b>100.0</b> (96.2-100.0)	<b>100.0</b> (87.7-100.0)	<b>99.0</b> (94.4-100.0)
2	51	0	1	198	250	<b>98.1</b> (89.7-100.0)	<b>100.0</b> (98.2-100.0)	<b>100.0</b> (93.0-100.0)	<b>99.5</b> (97.2-100.0)
<b>Total</b>	<b>79</b>	<b>0</b>	<b>2</b>	<b>295</b>	<b>376</b>	<b>97.5</b> (91.4-99.7)	<b>100.0</b> (98.8-100.0)	<b>100.0</b> (95.4-100.0)	<b>99.3</b> (97.6-99.9)

Table 12 Clinical evaluations and global results. True positive (TP), false positive (FP), false negative (FN), true negative values (TN), and total samples (N). Sensitivity (Sens), specificity (Spec), Predictive Positive Values (PPV), Predictive Negative Values (NPV), 95% interval of confidence (CI) for CerTest C. difficile GDH+Toxin A+B+Lactoferrin (Toxin A).

**For Strip C: Clostridium difficile Toxin B**

Ref.	Location	Sample type	Reference	Target
1	Hospital Clínico Lozano Blesa (Zaragoza, Spain) April-June 2017	Human stool samples	Code (17.1.1.)	<i>Clostridium difficile</i> Toxin B
2	Hospital Clínico Lozano Blesa (Zaragoza, Spain) April-May 2021	Human stool samples	Code (17.1.2)	<i>Clostridium difficile</i> Toxin B

Table 13 Resume of Clinical evaluations. Location, sample type, target.

Ref.	TP	FP	FN	TN	N	Sens. (%) CI (95%)	Spec. (%) CI (%)	PPV (%) CI (%)	NPV (%) CI (%)
1	36	1	0	89	126	<b>100.0</b> (90.3-100.0)	<b>98.9</b> (94.0-100.0)	<b>97.3</b> (85.8-99.9)	<b>100.0</b> (95.9-100.0)
2	40	0	4	206	250	<b>90.9</b> (78.3-97.5)	<b>100.0</b> (98.2-100.0)	<b>100.0</b> (91.2-100.0)	<b>98.1</b> (95.2-99.5)
<b>Total</b>	<b>76</b>	<b>1</b>	<b>4</b>	<b>295</b>	<b>376</b>	<b>95.0</b> (87.7-98.6)	<b>99.7</b> (98.1-100.0)	<b>98.7</b> (93.0-100.0)	<b>98.7</b> (96.6-99.6)

Table 14 Clinical evaluations and global results. True positive (TP), false positive (FP), false negative (FN), true negative values (TN), and total samples (N). Sensitivity (Sens), specificity (Spec), Predictive Positive Values (PPV), Predictive Negative Values (NPV), 95% interval of confidence (CI) for CerTest C. difficile GDH+Toxin A+B+Lactoferrin (Toxin B).

**C. Hook Effect**

**CerTest C. difficile GDH+Toxin A+B+Lactoferrin** does not show inhibitory hook effect at concentration higher than 10<sup>5</sup>xLoD for *Clostridium difficile* antigen GDH, higher than 10<sup>2</sup>xLoD for *C. difficile* Toxin A, higher than 10<sup>4</sup>xLoD for *C. difficile* Toxin B and higher than 10<sup>3</sup>xLoD for human lactoferrin.

**D. Cross Reactivity**

An evaluation was performed to determine the cross reactivity of **CerTest C. difficile GDH+Toxin A+B+Lactoferrin**, no cross reactivity against organism, pathogens, substances that could cause infections:

**For Strip A: Clostridium difficile antigen GDH**

Cross reactivity										
<b>Virus</b>	Adenovirus	-	Coronavirus	-	Influenza B	-	Norovirus GII	-	Rotavirus	-
	Astrovirus	-	Influenza A	-	Norovirus GI	-	Respiratory Syncytial Virus	-		
<b>Bacteria</b>	<i>Campylobacter coli</i>	-	<i>Clostridium Haemolyticum</i>	-	<i>Helicobacter pylori</i>	-	<i>Salmonella typhimurium</i>	-	<i>Streptococcus pyogenes</i>	-
	<i>Campylobacter jejuni</i>	-	<i>Clostridium Novyi</i>	-	<i>Legionella pneumophila</i>	-	<i>Shigella boydii</i>	-	<i>Yersinia enterocolitica O:3</i>	-
	<i>Clostridium difficile Toxin A</i>	-	<i>Clostridium Tetani</i>	-	<i>Listeria monocytogenes</i>	-	<i>Shigella dysenteriae</i>	-	<i>Yersinia enterocolitica O:9</i>	-
	<i>Clostridium difficile Toxin B</i>	-	<i>Clostridium Septicum</i>	-	<i>Peptostreptococcus anaerobius</i>	-	<i>Shigella flexneri</i>	-		-
	<i>Clostridium perfringens</i>	-	<i>Escherichia coli O:157</i>	-	<i>Salmonella enteritidis</i>	-	<i>Shigella sonnei</i>	-		-
	<i>Clostridium bifementans</i>	-	<i>Escherichia coli O:026</i>	-	<i>Salmonella paratyphi</i>	-	<i>Staphylococcus aureus</i>	-		-
	<i>Clostridium Butyricum</i>	-	<i>Escherichia coli O:111</i>	-	<i>Salmonella typhi</i>	-	<i>Streptococcus pneumococcal</i>	-		-
<b>Parasites</b>	<i>Cryptosporidium Parvum</i>	-	<i>Entamoeba histolytica</i>	-	<i>Giardia lamblia</i>	-				
<b>Others</b>	Pig haemoglobin	-	Bovine haemoglobin	-	Bovine Lactoferrin	-	Bovine Transferrin	-		

Table 15 Cross reactivity results (GDH).

A specificity assay was performed for **CerTest Clostridium difficile antigen GDH**. This concluded that the test, could detect the following antigens: *C. sporogenes* (CECT 485) and *C. botulinum* (CECT 551).

In this specificity assay, *Clostridium sordellii* (ATCC 9714) was evaluated and no cross reactivity was found. In few intercomparison studies **CerTest Clostridium difficile antigen GDH** have a positive signal with this pathogen, but the nature of this positive result could not be established.

#### **For Strip B: Clostridium difficile Toxin A**

Cross reactivity										
<b>Virus</b>	Adenovirus	-	Coronavirus	-	Influenza B	-	Norovirus GII	-	Rotavirus	-
	Astrovirus	-	Influenza A	-	Norovirus GI	-	Respiratory Syncytial Virus	-		
<b>Bacteria</b>	<i>Campylobacter coli</i>	-	<i>Escherichia coli O:157</i>	-	<i>Listeria monocytogenes</i>	-	<i>Shigella boydii</i>	-	<i>Streptococcus pneumococcal</i>	-
	<i>Campylobacter jejuni</i>	-	<i>Escherichia coli O:026</i>	-	<i>Salmonella enteritidis</i>	-	<i>Shigella dysenteriae</i>	-	<i>Streptococcus pyogenes</i>	-
	<i>Clostridium difficile antigen GDH</i>	-	<i>Escherichia coli O:111</i>	-	<i>Salmonella paratyphi</i>	-	<i>Shigella flexneri</i>	-	<i>Yersinia enterocolitica O:3</i>	-
	<i>Clostridium difficile Toxin B</i>	-	<i>Helicobacter pylori</i>	-	<i>Salmonella typhi</i>	-	<i>Shigella sonnei</i>	-	<i>Yersinia enterocolitica O:9</i>	-
	<i>Clostridium perfringens</i>	-	<i>Legionella pneumophila</i>	-	<i>Salmonella typhimurium</i>	-	<i>Staphylococcus aureus</i>	-		-
<b>Parasites</b>	<i>Cryptosporidium Parvum</i>	-	<i>Entamoeba histolytica</i>	-	<i>Entamoeba dispar</i>	-	<i>Giardia lamblia</i>	-		
<b>Others</b>	Pig haemoglobin	-	Bovine haemoglobin	-						

Table 16 Cross reactivity results (Toxin A).

**For Strip C: Clostridium difficile Toxin B**

Cross reactivity										
<b>Virus</b>	Adenovirus	-	Coronavirus	-	Influenza B	-	Norovirus GII	-	Rotavirus	-
	Astrovirus	-	Influenza A	-	Norovirus GI	-	Respiratory Syncytial Virus	-		
<b>Bacteria</b>	Campylobacter coli	-	Escherichia coli O:157	-	Listeria monocytogenes	-	Shigella boydii	-	Streptococcus pneumococcal	-
	Campylobacter jejuni	-	Escherichia coli O:026	-	Salmonella enteritidis	-	Shigella dysenteriae	-	Streptococcus pyogenes	-
	Clostridium difficile antigen GDH	-	Escherichia coli O:111	-	Salmonella paratyphi	-	Shigella flexneri	-	Yersinia enterocolitica O:3	-
	Clostridium difficile Toxin A	-	Helicobacter pylori	-	Salmonella typhi	-	Shigella sonnei	-	Yersinia enterocolitica O:9	-
	Clostridium perfringens	-	Legionella pneumophila	-	Salmonella typhimurium	-	Staphylococcus aureus	-		
<b>Parasites</b>	Cryptosporidium Parvum	-	Entamoeba histolytica	-	Entamoeba dispar	-	Giardia lamblia	-		
<b>Others</b>	Pig haemoglobin	-	Bovine haemoglobin	-						

Table 17 Cross reactivity results (Toxin B).

**For Strip D: Human Lactoferrin**

Cross reactivity										
<b>Virus</b>	Adenovirus	-	Coronavirus	-	Influenza B	-	Norovirus GII	-	Rotavirus	-
	Astrovirus	-	Influenza A	-	Norovirus GI	-	Respiratory Syncytial Virus	-		
<b>Bacteria</b>	Campylobacter coli	-	Clostridium perfringens	-	Legionella pneumophila	-	Salmonella typhimurium	-	Staphylococcus aureus	-
	Campylobacter jejuni	-	Escherichia coli O:157	-	Listeria monocytogenes	-	Shigella boydii	-	Streptococcus pneumococcal	-
	Clostridium difficile antigen GDH	-	Escherichia coli O:026	-	Salmonella enteritidis	-	Shigella dysenteriae	-	Streptococcus pyogenes	-
	Clostridium difficile Toxin A	-	Escherichia coli O:111	-	Salmonella paratyphi	-	Shigella flexneri	-	Yersinia enterocolitica O:3	-
	Clostridium difficile Toxin B	-	Helicobacter pylori	-	Salmonella typhi	-	Shigella sonnei	-	Yersinia enterocolitica O:9	-
<b>Parasites</b>	Cryptosporidium Parvum	-	Entamoeba histolytica	-	Entamoeba dispar	-	Giardia lamblia	-		
<b>Others</b>	Pig haemoglobin	-	Bovine haemoglobin	-	Bovine lactoferrin	-	Bovine transferrin	-		

Table 18 Cross reactivity results (Lactoferrin).

**E. Interferences**

An evaluation was performed to determine the possible interferences of **CerTest C. difficile GDH+Toxin A+B+Lactoferrin**; no interferences against the substances tested were detected:

**For Strip A: Clostridium difficile antigen GDH**

	Potencial interfering substance	Concentration	Results	Potencial interfering substance	Concentration	Results
<b>Exogenous</b>	Metronidazole	3.0 mg/mL	No interference	Loperamide hydrochloride (Fortasec)	0.15 mg/mL	No interference
	Ampicillin	3.0 mg/mL	No interference	Heparin (Hibor)	350.0 IU/mL	No interference
	Oseltamivir	0.3 mg/mL	No interference	Almagato (Almax)	3.0 mg/mL	No interference
	Amantadine	0.3 mg/mL	No interference	Fosfamycin (Monurol)	3.0 · 10 <sup>-3</sup> mg/mL	No interference
	Ribavirin	3.0 mg/mL	No interference	Acetylcysteine (Fluimucil)	3.0 mg/mL	No interference
	Codeine (Toseina)	0.2 mg/mL	No interference	Dexketoprofen trometamol (Enantyum)	0.3 mg/mL	No interference
	Benzocaine (Angileptol)	0.3 mg/mL	No interference	Levofloxacin	3.0 mg/mL	No interference
	Cloperastine (Flutox)	0.3 mg/mL	No interference	Ciprofloxacin	0.3 mg/mL	No interference
	Carbocisteine (Iniston mucolítico)	3.0 mg/mL	No interference	Rifampicin (Rifaldin)	0.3 mg/mL	No interference
	Loratadine	0.3 mg/mL	No interference	Phenoxymethylpenicillin potassium	3.0 mg/mL	No interference
	Dexchlorpheniramine (Polaramine)	0.3 mg/mL	No interference	Ambroxol hydrochloride (Mucosan)	0.3 mg/mL	No interference
	Ebastine (Ebastel)	3.0 mg/mL	No interference	Macrogol 3350 (Movicol)	3.0 mg/mL	No interference
	Acetyl Salicylic (Adiro)	0.3 mg/mL	No interference	Lysine Carbocysteinate (Pectox)	3.0 · 10 <sup>-2</sup> mg/mL	No interference
	Ibuprofen (Espidifen)	3.0 mg/mL	No interference	Hydroxyzine dihydrochloride	0.3 mg/mL	No interference
	Paracetamol (Dolocatil)	5.0 mg/mL	No interference	Lorazepam	3.0 · 10 <sup>-3</sup> mg/mL	No interference
	Metamizole (Nolotil)	5.0 mg/mL	No interference	Amoxicillin	3.0 mg/mL	No interference
	Prednisone	0.3 mg/mL	No interference	Mercaptopurine	0.3 mg/mL	No interference
	Omeprazole	2.0 · 10 <sup>-3</sup> mg/mL	No interference	Biotine	0.10 mg/mL	No interference
	Naso GEL	0.9 mg/mL	No interference	Sore Throat Phenol spray	0.5 mg/mL	No interference
	CVS Nasal Spray (Cromolyn)	4 mg/mL	No interference	Tobramycin	0.3 mg/mL	No interference
Afrin (Oxymetazoline)	5.0 · 10 <sup>-2</sup> mg/mL	No interference	Mupirocin	2.5 · 10 <sup>-2</sup> mg/mL	No interference	
CVS Nasal Drops (Phenylephrine)	10.0 mg/mL	No interference	Fluticasone Propionate	5.0 · 10 <sup>-2</sup> mg/mL	No interference	
ZICAM	1.0 mg/mL	No interference	Homeopathic	DIL 1/10	No interference	
Phenylpropanolamine	0.3 mg/mL	No interference				
<b>Endogenous</b>	Human haemoglobin	5.0 µg/mL	No interference	Human lactoferrin	5.0 µg/mL	No interference
	Human transferrin	0.5 µg/mL	No interference	Mucine	0.5 %	No interference
	Human calprotectin	5.0 µg/mL	No interference	Human blood	5.0 %	No interference

Table 19 Clostridium difficile antigen GDH interferences results.

**For Strip B: Clostridium difficile Toxin A**

	Potencial interfering substance	Concentration	Results	Potencial interfering substance	Concentration	Results
<b>Exogenous</b>	Metronidazole	3.0 mg/mL	No interference	Loperamide hydrochloride ( <i>Fortasec</i> )	0.15 mg/mL	No interference
	Ampicillin	3.0 mg/mL	No interference	Heparin ( <i>Hibor</i> )	350.0 IU/mL	No interference
	Oseltamivir	0.3 mg/mL	No interference	Almagato ( <i>Almax</i> )	3.0 mg/mL	No interference
	Amantadine	0.3 mg/mL	No interference	Fosfamycin ( <i>Monurol</i> )	3.0·10 <sup>-3</sup> mg/mL	No interference
	Ribavirin	3.0 mg/mL	No interference	Acetylcysteine ( <i>Fluimucil</i> )	3.0 mg/mL	No interference
	Codeine ( <i>Toseina</i> )	0.2 mg/mL	No interference	Dexketoprofen trometamol ( <i>Enantyum</i> )	0.3 mg/mL	No interference
	Benzocaine ( <i>Angileptol</i> )	0.3 mg/mL	No interference	Levofloxacin	3.0 mg/mL	No interference
	Cloperastine ( <i>Flutox</i> )	0.3 mg/mL	No interference	Ciprofloxacin	0.3 mg/mL	No interference
	Carbocisteine ( <i>Iniston mucolítico</i> )	3.0 mg/mL	No interference	Rifampicin ( <i>Rifaldin</i> )	0.3 mg/mL	No interference
	Loratadine	0.3 mg/mL	No interference	Phenoxymethylpenicillin potassium	3.0 mg/mL	No interference
	Dexchlorpheniramine ( <i>Polaramine</i> )	0.3 mg/mL	No interference	Ambroxol hydrochloride ( <i>Mucosan</i> )	0.3 mg/mL	No interference
	Ebastine ( <i>Ebastel</i> )	3.0 mg/mL	No interference	Macrogol 3350 ( <i>Movicol</i> )	3.0 mg/mL	No interference
	Acetyl Salicylic ( <i>Adiro</i> )	0.3 mg/mL	No interference	Lysine Carbocysteinate ( <i>Pectox</i> )	3.0·10 <sup>-2</sup> mg/mL	No interference
	Ibuprofen ( <i>Espidifen</i> )	3.0 mg/mL	No interference	Hydroxyzine dihydrochloride	0.3 mg/mL	No interference
	Paracetamol ( <i>Dolocatil</i> )	5.0 mg/mL	No interference	Lorazepam	3.0·10 <sup>-3</sup> mg/mL	No interference
	Metamizole ( <i>Nolotil</i> )	5.0 mg/mL	No interference	Amoxicillin	3.0 mg/mL	No interference
	Prednisone	0.3 mg/mL	No interference	Mercaptopurine	0.3 mg/mL	No interference
	Omeprazole	2.0·10 <sup>-3</sup> mg/mL	No interference	Biotine	0.10 mg/mL	No interference
	Naso GEL	0.9 mg/mL	No interference	Sore Throat Phenol spray	0.5 mg/mL	No interference
	CVS Nasal Spray ( <i>Cromolyn</i> )	4.0 mg/mL	No interference	Tobramycin	0.3 mg/mL	No interference
Afrin ( <i>Oxymetazoline</i> )	5.0·10 <sup>-2</sup> mg/mL	No interference	Mupirocin	2.5·10 <sup>-2</sup> mg/mL	No interference	
CVS Nasal Drops ( <i>Phenylephrine</i> )	10.0 mg/mL	No interference	Fluticasone Propionate	5.0·10 <sup>-2</sup> mg/mL	No interference	
ZICAM	1.0 mg/mL	No interference	Homeopathic	DIL 1/10	No interference	
Phenylpropanolamine	0.3 mg/mL	No interference				
<b>Endogenous</b>	Human haemoglobin	5.0 µg/mL	No interference	Human lactoferrin	5.0 µg/mL	No interference
	Human transferrin	0.5 µg/mL	No interference	Mucine	0.5 %	No interference
	Human calprotectin	5.0 µg/mL	No interference	Human blood	5.0 %	No interference

Table 20 Clostridium difficile toxin A interferences results.

**For Strip C: Clostridium difficile Toxin B**

	Potencial interfering substance	Concentration	Results	Potencial interfering substance	Concentration	Results
<b>Exogenous</b>	Metronidazole	3.0 mg/mL	No interference	Loperamide hydrochloride ( <i>Fortasec</i> )	0.15 mg/mL	No interference
	Ampicillin	3.0 mg/mL	No interference	Heparin ( <i>Hibor</i> )	350.0 IU/mL	No interference
	Oseltamivir	0.3 mg/mL	No interference	Almagato ( <i>Almax</i> )	3.0 mg/mL	No interference
	Amantadine	0.3 mg/mL	No interference	Fosfamycin ( <i>Monurol</i> )	3.0·10 <sup>-3</sup> mg/mL	No interference
	Ribavirin	3.0 mg/mL	No interference	Acetylcysteine ( <i>Fluimucil</i> )	3.0 mg/mL	No interference
	Codeine ( <i>Toseina</i> )	0.2 mg/mL	No interference	Dexketoprofen trometamol ( <i>Enantyum</i> )	0.3 mg/mL	No interference
	Benzocaine ( <i>Angileptol</i> )	0.3 mg/mL	No interference	Levofloxacin	3.0 mg/mL	No interference
	Cloperastine ( <i>Flutox</i> )	0.3 mg/mL	No interference	Ciprofloxacin	0.3 mg/mL	No interference
	Carbocisteine ( <i>Iniston mucolítico</i> )	3.0 mg/mL	No interference	Rifampicin ( <i>Rifaldin</i> )	0.3 mg/mL	No interference
	Loratadine	0.3 mg/mL	No interference	Phenoxymethylpenicillin potassium	3.0 mg/mL	No interference
	Dexchlorpheniramine ( <i>Polaramine</i> )	0.3 mg/mL	No interference	Ambroxol hydrochloride ( <i>Mucosan</i> )	0.3 mg/mL	No interference
	Ebastine ( <i>Ebastel</i> )	3.0 mg/mL	No interference	Macrogol 3350 ( <i>Movicol</i> )	3.0 mg/mL	No interference
	Acetyl Salicylic ( <i>Adiro</i> )	0.3 mg/mL	No interference	Lysine Carbocysteinate ( <i>Pectox</i> )	3.0·10 <sup>-2</sup> mg/mL	No interference
	Ibuprofen ( <i>Espidifen</i> )	3.0 mg/mL	No interference	Hydroxyzine dihydrochloride	0.3 mg/mL	No interference
	Paracetamol ( <i>Dolocatil</i> )	5.0 mg/mL	No interference	Lorazepam	3.0·10 <sup>-3</sup> mg/mL	No interference
	Metamizole ( <i>Nolotil</i> )	5.0 mg/mL	No interference	Amoxicillin	3.0 mg/mL	No interference
	Prednisone	0.3 mg/mL	No interference	Mercaptopurine	0.3 mg/mL	No interference
	Omeprazole	2.0·10 <sup>-3</sup> mg/mL	No interference	Biotine	0.10 mg/mL	No interference
	Naso GEL	0.9 mg/mL	No interference	Sore Throat Phenol spray	0.5 mg/mL	No interference
	CVS Nasal Spray ( <i>Cromolyn</i> )	4.0 mg/mL	No interference	Tobramycin	0.3 mg/mL	No interference
Afrin ( <i>Oxymetazoline</i> )	5.0·10 <sup>-2</sup> mg/mL	No interference	Mupirocin	2.5·10 <sup>-2</sup> mg/mL	No interference	
CVS Nasal Drops ( <i>Phenylephrine</i> )	10.0 mg/mL	No interference	Fluticasone Propionate	5.0·10 <sup>-2</sup> mg/mL	No interference	
ZICAM	1.0 mg/mL	No interference	Homeopathic	DIL 1/10	No interference	
Phenylpropanolamine	0.3 mg/mL	No interference				
<b>Endogenous</b>	Human haemoglobin	5.0 µg/mL	No interference	Human lactoferrin	5.0 µg/mL	No interference
	Human transferrin	0.5 µg/mL	No interference	Mucine	0.5 %	No interference
	Human calprotectin	5.0 µg/mL	No interference	Human blood	5.0 %	No interference

Table 21 Clostridium difficile toxin B interferences results.

**For Strip D: Human Lactoferrin**

	Potencial interfering substance	Concentration	Results	Potencial interfering substance	Concentration	Results
<b>Exogenous</b>	Metronidazole	3.0 mg/mL	No interference	Loperamide hydrochloride ( <i>Fortasec</i> )	0.15 mg/mL	No interference
	Ampicillin	3.0 mg/mL	No interference	Heparin ( <i>Hibor</i> )	350.0 IU/mL	No interference
	Oseltamivir	0.3 mg/mL	No interference	Almagato ( <i>Almax</i> )	3.0 mg/mL	No interference
	Amantadine	0.3 mg/mL	No interference	Fosfamycin ( <i>Monurol</i> )	3.0·10 <sup>-3</sup> mg/mL	No interference
	Ribavirin	3.0 mg/mL	No interference	Acetylcysteine ( <i>Fluimucil</i> )	3.0 mg/mL	No interference
	Codeine ( <i>Toseina</i> )	0.2 mg/mL	No interference	Dexketoprofen trometamol ( <i>Enantyum</i> )	0.3 mg/mL	No interference
	Benzocaine ( <i>Angileptol</i> )	0.3 mg/mL	No interference	Levofloxacin	3.0 mg/mL	No interference
	Cloperastine ( <i>Flutox</i> )	0.3 mg/mL	No interference	Ciprofloxacin	0.3 mg/mL	No interference
	Carbocisteine ( <i>Iniston mucolítico</i> )	3.0 mg/mL	No interference	Rifampicin ( <i>Rifaldin</i> )	0.3 mg/mL	No interference
	Loratadine	0.3 mg/mL	No interference	Phenoxymethylpenicillin potassium	3.0 mg/mL	No interference
	Dexchlorpheniramine ( <i>Polaramine</i> )	0.3 mg/mL	No interference	Ambroxol hydrochloride ( <i>Mucosan</i> )	0.3 mg/mL	No interference
	Ebastine ( <i>Ebastel</i> )	3.0 mg/mL	No interference	Macrogol 3350 ( <i>Movicol</i> )	3.0 mg/mL	No interference
	Acetyl Salicylic ( <i>Adiro</i> )	0.3 mg/mL	No interference	Lysine Carbocysteinate ( <i>Pectox</i> )	3.0·10 <sup>-2</sup> mg/mL	No interference
	Ibuprofen ( <i>Espidifen</i> )	3.0 mg/mL	No interference	Hydroxyzine dihydrochloride	0.3 mg/mL	No interference
	Paracetamol ( <i>Dolocatil</i> )	5.0 mg/mL	No interference	Lorazepam	3.0·10 <sup>-3</sup> mg/mL	No interference
	Metamizole ( <i>Nolotil</i> )	5.0 mg/mL	No interference	Amoxicillin	3.0 mg/mL	No interference
	Prednisone	0.3 mg/mL	No interference	Mercaptopurine	0.3 mg/mL	No interference
	Omeprazole	2.0·10 <sup>-3</sup> mg/mL	No interference	Biotine	0.10 mg/mL	No interference
	Naso GEL	0.9 mg/mL	No interference	Sore Throat Phenol spray	0.5 mg/mL	No interference
	CVS Nasal Spray ( <i>Cromolyn</i> )	4.0 mg/mL	No interference	Tobramycin	0.3 mg/mL	No interference
Afrin ( <i>Oxymetazoline</i> )	5.0·10 <sup>-2</sup> mg/mL	No interference	Mupirocin	2.5·10 <sup>-2</sup> mg/mL	No interference	
CVS Nasal Drops ( <i>Phenylephrine</i> )	10.0 mg/mL	No interference	Fluticasone Propionate	5.0·10 <sup>-2</sup> mg/mL	No interference	
ZICAM	1.0 mg/mL	No interference	Homeopathic	DIL 1/10	No interference	
Phenylpropanolamine	0.3 mg/mL	No interference				
<b>Endogenous</b>	Human haemoglobin	5.0 µg/mL	No interference	Human lactoferrin	5.0 µg/mL	No interference
	Human transferrin	0.5 µg/mL	No interference	Mucine	0.5 %	No interference
	Human calprotectin	5.0 µg/mL	No interference	Human blood	5.0 %	No interference

Table 22 Human Lactoferrin interferences results.

---

**F. Repeatability and Reproducibility**

Following the study of repeatability and reproducibility of **CerTest C. difficile GDH+Toxin A+B+Lactoferrin**, performed using different internal samples, negative and positive, no significative differences have been observed within the evaluations.

For information purposes only

## ANNEX 1

## KIT 20 CARD C. DIFFICILE GDH+TOXIN A+B+LACTOFERRIN. Catalogue reference: LX862001VF

Annex for the following reference:

PRODUCT	REFERENCE
<b>CerTest C. difficile GDH+Toxin A+B+Lactoferrin</b> Combo Card (20 tests kit with stool vials)	LX862001VF

Table A 1. Reference for kit 20 combo card of CerTest C.difficile GDH+Toxin A+B+Lactoferrin.

### I. Materials

MATERIALS PROVIDED	MATERIALS REQUIRED BUT NO PROVIDED
<ul style="list-style-type: none"> <li>- 20x <b>CerTest C. difficile GDH+Toxin A+B+Lactoferrin</b> (combo card test format)</li> <li>- 20x <b>GDH+Toxin A+B stool collection tubes</b> with diluent</li> <li>- 20x <b>Lactoferrin stool collection tubes</b> with diluent</li> <li>- 1x IFU Instructions for use (including Quick guide use).</li> </ul>	<ul style="list-style-type: none"> <li>- Disposable gloves (other personal protective equipment that will be considered necessary)</li> <li>- Timer</li> <li>- Specimen collection container</li> </ul>

Table A 1.1 Materials necessities for test performance.

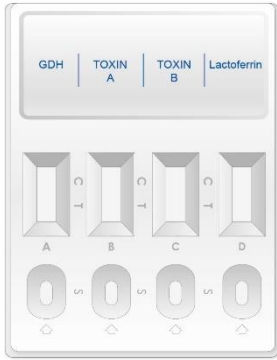
### II. Test procedures (samples)

Follow proper infection control practices. Allow tests and samples to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.


1. Proceed to shake the **stool collection tubes** (both types) in order to assure good sample dispersion.
2. Remove **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (combo card test) from its sealed bag just before using it.
3. Take the **GDH+Toxin A+B stool collection tube**, cut the end of the cap (1) and dispense 3 drops in the circular window marked with the letter A (2), 3 drops, using the same tube, in the circular window marked with letter B (3) and 3 drops, using the same tube, in the circular window marked with letter C (4). Avoid adding solid particles with the liquid.
4. Take the **Lactoferrin stool collection tube**, open the cap 2 (5) and dispense 3 drops in the circular window marked with the letter D (6). Avoid adding solid particles with the liquid.

5. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.


If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of diluent until seeing the liquid running through the reaction zone.

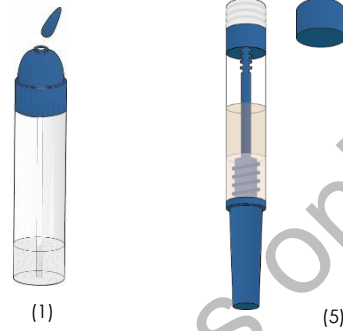
Results window  
 ← Control Line (C)  
 ← Test Line (T)  
 Sample (S) circular window

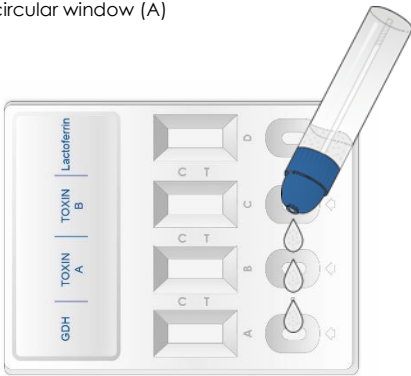
(2) Add 3 drops in the circular window (A)

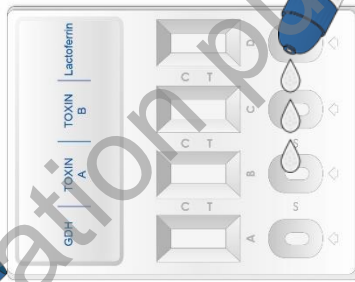
(3) Add 3 drops in the circular window (B)

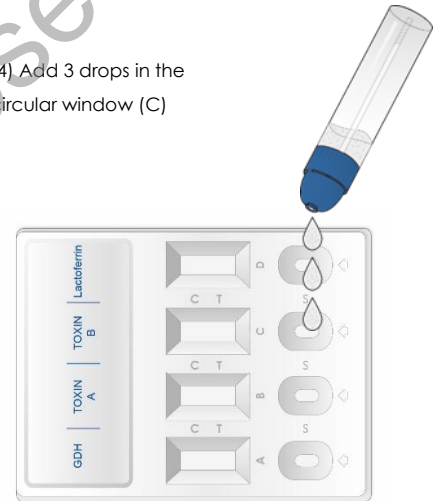(4) Add 3 drops in the circular window (C)





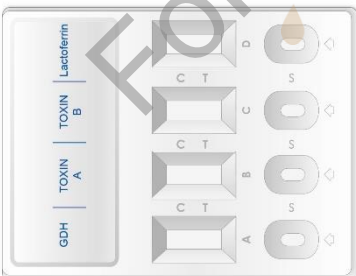
(6) Add 3 drops in the circular window (D)







**Wait 10 minutes**

### III. Interpretation of results

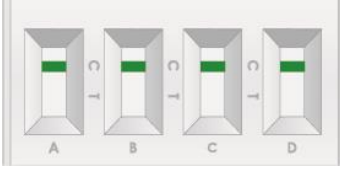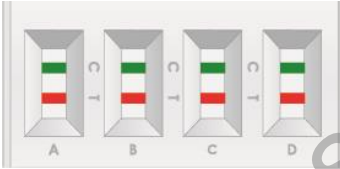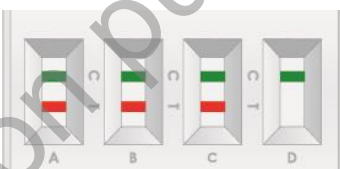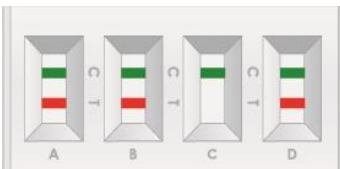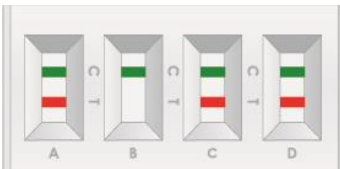(please, refer to the illustration below)

	GDH	Toxin A	Toxin B	Lactoferrin		Interpretation of the results
1.	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE		<p>There are no GDH, Toxin A or Toxin B of <i>C. difficile</i> presence and neither human lactoferrin. No infection caused by <i>Clostridium difficile</i>.</p> <p>A negative result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN	GREEN	GREEN	GREEN		
2.	POSITIVE	POSITIVE	POSITIVE	POSITIVE		<p>There are GDH, Toxin A, Toxin B and Lactoferrin presence. Possible infection caused by <i>Clostridium difficile</i> and there is an intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN-RED		
3.	POSITIVE	POSITIVE	POSITIVE	NEGATIVE		<p>There is GDH, Toxin A and Toxin B presence. Possible infection caused by <i>Clostridium difficile</i>.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN		
4.	POSITIVE	POSITIVE	NEGATIVE	POSITIVE		<p>There is GDH, Toxin A and Lactoferrin presence. Possible infection caused by <i>Clostridium difficile</i>, and there is an intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN-RED	GREEN	GREEN-RED		
5.	POSITIVE	NEGATIVE	POSITIVE	POSITIVE		<p>There is GDH, Toxin B and Lactoferrin presence. Possible infection caused by <i>Clostridium difficile</i>, and there is an intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN	GREEN-RED	GREEN-RED		

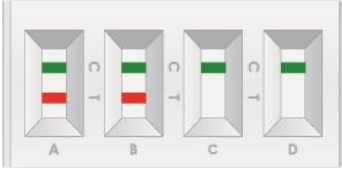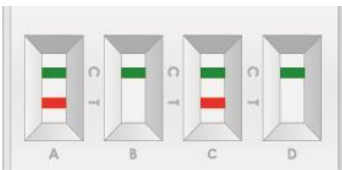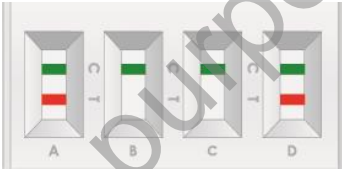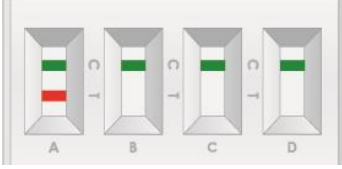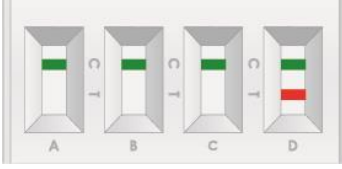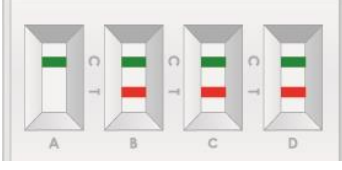	GDH	Toxin A	Toxin B	Lactoferrin		Interpretation of the results
6.	POSITIVE	POSITIVE	NEGATIVE	NEGATIVE		<p>There is GDH and Toxin A presence. Possible infection caused by <i>Clostridium difficile</i>.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN-RED	GREEN	GREEN		
7.	POSITIVE	NEGATIVE	POSITIVE	NEGATIVE		<p>There is GDH and Toxin B presence. Possible infection caused by <i>Clostridium difficile</i>.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN	GREEN-RED	GREEN		
8.	POSITIVE	NEGATIVE	NEGATIVE	POSITIVE		<p>There is GDH and Lactoferrin presence. Possible infection caused by <i>Clostridium difficile</i> and there is an intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN	GREEN	GREEN-RED		
9.	POSITIVE	NEGATIVE	NEGATIVE	NEGATIVE		<p>There is GDH presence. Possible infection caused by <i>Clostridium difficile</i>.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN	GREEN	GREEN		
10.	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE		<p>There is Lactoferrin presence. Possible intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN	GREEN	GREEN	GREEN-RED		
11.	NEGATIVE	POSITIVE	POSITIVE	POSITIVE		<p>There is Toxin A, Toxin B and Lactoferrin presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN	GREEN	GREEN-RED		

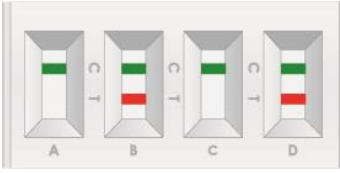
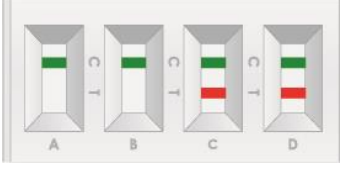
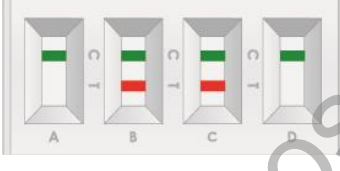
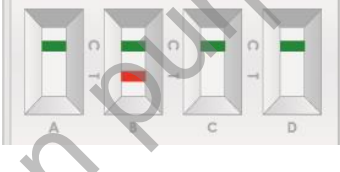
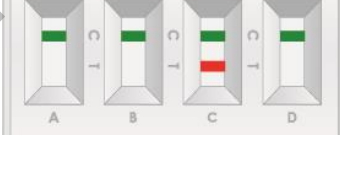
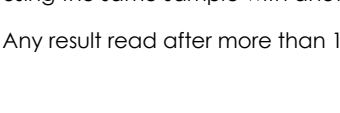
	GDH	Toxin A	Toxin B	Lactoferrin		Interpretation of the results
12.	<b>NEGATIVE</b>	<b>POSITIVE</b>	<b>NEGATIVE</b>	<b>POSITIVE</b>		There is Toxin A and Lactoferrin presence. If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.
	GREEN	GREEN-RED	GREEN	GREEN-RED		
13.	<b>NEGATIVE</b>	<b>NEGATIVE</b>	<b>POSITIVE</b>	<b>POSITIVE</b>		There is Toxin B and Lactoferrin presence. If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.
	GREEN	GREEN	GREEN-RED	GREEN-RED		
14.	<b>NEGATIVE</b>	<b>POSITIVE</b>	<b>POSITIVE</b>	<b>NEGATIVE</b>		There is Toxin A and Toxin B presence. If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.
	GREEN	GREEN-RED	GREEN-RED	GREEN		
15.	<b>NEGATIVE</b>	<b>POSITIVE</b>	<b>NEGATIVE</b>	<b>NEGATIVE</b>		There is Toxin A presence. If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.
	GREEN	GREEN-RED	GREEN	GREEN		
16.	<b>NEGATIVE</b>	<b>NEGATIVE</b>	<b>POSITIVE</b>	<b>NEGATIVE</b>		There is Toxin B presence. If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.
	GREEN	GREEN	GREEN-RED	GREEN		
17.	<b>INVALID</b>					Invalid result either A, B, C or D, we recommend repeating the assay using the same sample with another test.  Any result read after more than 10 minutes will be considered invalid.
	Any other results					

Table A 1.2. CerTest C. difficile GDH+Toxin A+B+Lactoferrin performance interpretation of the results.

**Whatever results detailed between 11 and 16, are not very common. A very low percentage of specimens might result negative for GDH but positive for toxins.**

**NEGATIVE:** Only one GREEN line appears across the results window in the site marked with the letter C (control line).

**POSITIVE:** In addition to the GREEN control line, a RED line also appears in the site marked with the letter T (test line) in the results window.

The result would be considered as positive independent on the intensity of the RED test line.

---

**INVALID:** Total absence of the control coloured line (GREEN) regardless the appearance or not of the test line (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persists, discontinue using the test kit and contact your local distributor.

#### **NOTES ON THE INTERPRETATION OF RESULTS**

The intensity of the red coloured band in the test line (T) in the results window will vary depending on the concentration of antigen presents in the specimen. However, neither the quantitative value nor the rate of increase in antigen can be determined by this qualitative test.

## **IV. Quality control**

#### INTERNAL CONTROL: control line

Internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

For information purposes only

## ANNEX 2

## KIT 20 CARD C. *DIFFICILE* GDH+TOXIN A+B+LACTOFERRIN. Catalogue reference: LX862001VFC

Annex for the following reference:

PRODUCT	REFERENCE
<b>CerTest C. difficile GDH+Toxin A+B+Lactoferrin</b> Combo Card (20 tests kit with stool vials + PC)	LX862001VFC

Table A 2. Reference for kit 20 combo card of CerTest C. *difficile* GDH+Toxin A+B+Lactoferrin with positive control.

### I. Materials

MATERIALS PROVIDED	MATERIALS REQUIRED BUT NO PROVIDED
<ul style="list-style-type: none"> <li>- 20x <b>CerTest C. difficile GDH+Toxin A+B+Lactoferrin</b> (combo card test format)</li> <li>- 20x <b>GDH+Toxin A+B stool collection tubes</b> with diluent</li> <li>- 20x <b>Lactoferrin stool collection tubes</b> with diluent</li> <li>-1x <b>CerTest C. difficile antigen GDH Positive Control</b></li> <li>-1x <b>CerTest C. difficile Toxin A Positive Control</b></li> <li>-1x <b>CerTest C. difficile Toxin B Positive Control</b></li> <li>-1x <b>CerTest Lactoferrin Positive Control</b></li> <li>-4x <b>Testing tube</b></li> <li>-4x <b>Disposable pipette</b></li> <li>-3x <b>Red cap reagent tube</b></li> <li>-1x <b>Double cap reagent tube</b></li> <li>-1x IFU Instructions for use (including Quick guide use).</li> </ul>	<ul style="list-style-type: none"> <li>- Disposable gloves (other personal protective equipment that will be considered necessary)</li> <li>- Timer</li> <li>- Specimen collection container</li> </ul>

Table A 2.1 Materials necessities for test performance.

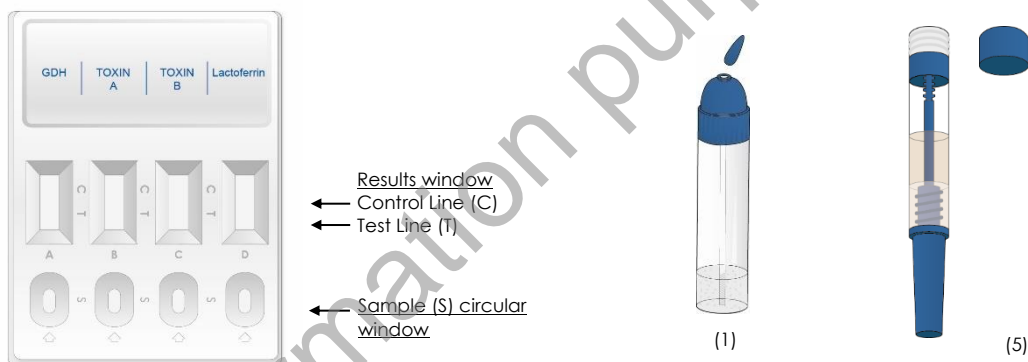
## II. Test procedures (samples)


Follow proper infection control practices. Allow tests and samples to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Proceed to shake the **stool collection tube** (both types) in order to assure good sample dispersion.
2. Remove **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (combo card test) from its sealed bag just before using it.
3. Take the **GDH+Toxin A+B stool collection tube**, cut the end of the cap (1) and dispense 3 drops in the circular window marked with the letter A (2), 3 drops, using the same tube, in the circular window marked with letter B (3) and 3 drops, using the same tube, in the circular window marked with letter C (4). Avoid adding solid particles with the liquid.
4. Take the **Lactoferrin stool collection tube**, open the cap 2 (5) and dispense 3 drops in the circular window marked with the letter D (6). Avoid adding solid particles with the liquid.
5. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.


If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of diluent until seeing the liquid running through the reaction zone.

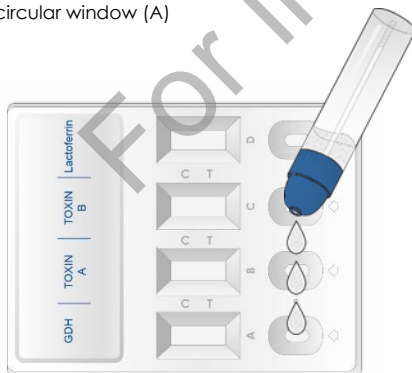
(2) Add 3 drops in the circular window (A)

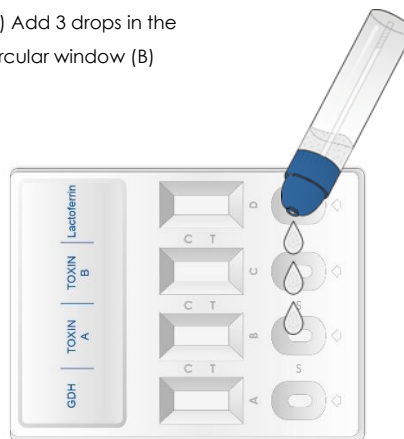
(3) Add 3 drops in the circular window (B)

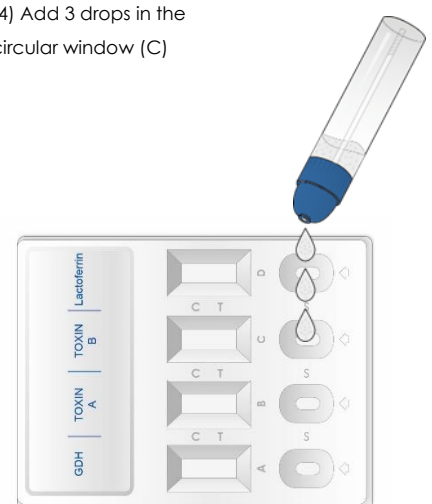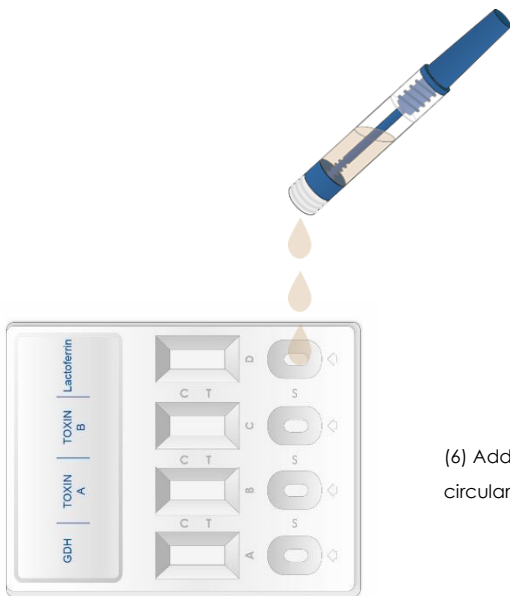(4) Add 3 drops in the circular window (C)

(6) Add 3 drops in the circular window (D)

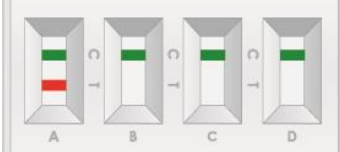
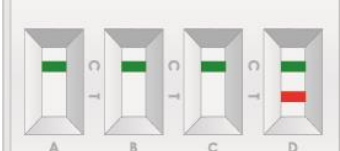
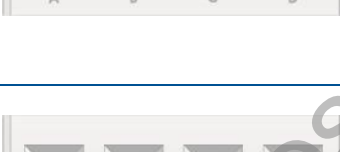
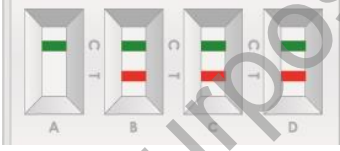
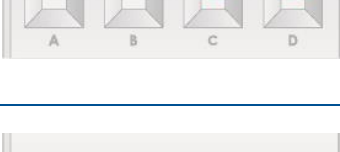
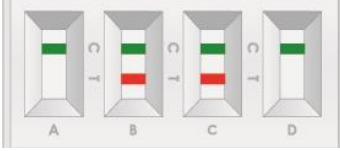
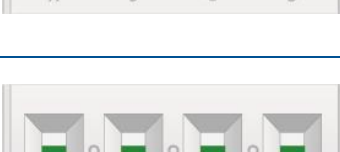
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
Wait 10 minutes

### III. Interpretation of results

(please, refer to the illustration below)

	GDH	Toxin A	Toxin B	Lactoferrin		Interpretation of the results
1.	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE		<p>There are no GDH, Toxin A or Toxin B of <i>C. difficile</i> presence and neither human lactoferrin. No infection caused by <i>Clostridium difficile</i>.</p> <p>A negative result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN	GREEN	GREEN	GREEN		
2.	POSITIVE	POSITIVE	POSITIVE	POSITIVE		<p>There are GDH, Toxin A, Toxin B and Lactoferrin presence. Possible infection caused by <i>Clostridium difficile</i> and there is an intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN-RED		
3.	POSITIVE	POSITIVE	POSITIVE	NEGATIVE		<p>There is GDH, Toxin A and Toxin B presence. Possible infection caused by <i>Clostridium difficile</i>.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN		

	GDH	Toxin A	Toxin B	Lactoferrin		Interpretation of the results
4.	POSITIVE	POSITIVE	NEGATIVE	POSITIVE		<p>There is GDH, Toxin A and Lactoferrin presence. Possible infection caused by <i>Clostridium difficile</i>, and there is an intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN-RED	GREEN	GREEN-RED		
5.	POSITIVE	NEGATIVE	POSITIVE	POSITIVE		<p>There is GDH, Toxin B and Lactoferrin presence. Possible infection caused by <i>Clostridium difficile</i>, and there is an intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN	GREEN-RED	GREEN-RED		
6.	POSITIVE	POSITIVE	NEGATIVE	NEGATIVE		<p>There is GDH and Toxin A presence. Possible infection caused by <i>Clostridium difficile</i>.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN-RED	GREEN	GREEN		
7.	POSITIVE	NEGATIVE	POSITIVE	NEGATIVE		<p>There is GDH and Toxin B presence. Possible infection caused by <i>Clostridium difficile</i>.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN	GREEN-RED	GREEN		
8.	POSITIVE	NEGATIVE	NEGATIVE	POSITIVE		<p>There is GDH and Lactoferrin presence. Possible infection caused by <i>Clostridium difficile</i> and there is an intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN	GREEN	GREEN-RED		
9.	POSITIVE	NEGATIVE	NEGATIVE	NEGATIVE		

	GDH	Toxin A	Toxin B	Lactoferrin		Interpretation of the results
	GREEN-RED	GREEN	GREEN	GREEN		<p>There is GDH presence. Possible infection caused by <i>Clostridium difficile</i>.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
10.	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE		<p>There is Lactoferrin presence. Possible intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN	GREEN	GREEN	GREEN-RED		<p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
11.	NEGATIVE	POSITIVE	POSITIVE	POSITIVE		<p>There is Toxin A, Toxin B and Lactoferrin presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN	GREEN	GREEN-RED		
12.	NEGATIVE	POSITIVE	NEGATIVE	POSITIVE		<p>There is Toxin A and Lactoferrin presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN-RED	GREEN	GREEN-RED		
13.	NEGATIVE	NEGATIVE	POSITIVE	POSITIVE		<p>There is Toxin B and Lactoferrin presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN	GREEN-RED	GREEN-RED		
14.	NEGATIVE	POSITIVE	POSITIVE	NEGATIVE		<p>There is Toxin A and Toxin B presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN-RED	GREEN-RED	GREEN		
15.	NEGATIVE	POSITIVE	NEGATIVE	NEGATIVE		<p>There is Toxin A presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN-RED	GREEN	GREEN		

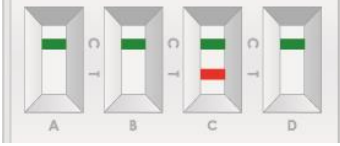
	GDH	Toxin A	Toxin B	Lactoferrin		Interpretation of the results
16.	NEGATIVE	NEGATIVE	POSITIVE	NEGATIVE		<p>There is Toxin B presence. If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN	GREEN-RED	GREEN		
17.	INVALID				<p>Invalid result either A, B, C or D we recommend repeating the assay using the same sample with another test.</p> <p>Any result read after more than 10 minutes will be considered invalid.</p>	
	Any other results					

Table A 2.2 CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin performance interpretation of the results.

Whatever results detailed between 11 and 16, are not very common. A very low percentage of specimens might result negative for GDH but positive for toxins.

**NEGATIVE:** Only one GREEN line appears across the results window in the site marked with the letter C (control line).

**POSITIVE:** In addition to the GREEN control line, a RED line also appears in the site marked with the letter T (test line) in the results window.

The result would be considered as positive independent on the intensity of the RED test line.

**INVALID:** Total absence of the control coloured line (GREEN) regardless the appearance or not of the test line (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persists, discontinue using the test kit and contact your local distributor.

#### NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red coloured band in the test line (T) in the results window will vary depending on the concentration of antigen presents in the specimen. However, neither the quantitative value nor the rate of increase in antigen can be determined by this qualitative test.

## IV. Quality control

**CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin** (combo card test) presents five controls:

INTERNAL CONTROL: control line

**Internal control:** internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

EXTERNAL CONTROL: POSITIVE CONTROL SWABS

**External control: CerTest C. difficile antigen GDH Positive Control, CerTest C. difficile Toxin A Positive Control, CerTest C. difficile Toxin B Positive Control and CerTest Lactoferrin Positive Control** (positive control swabs).

#### A. External positive controls

It is recommended to use the same device for both external positive controls.

##### 1. CerTest C. difficile antigen GDH Positive Control

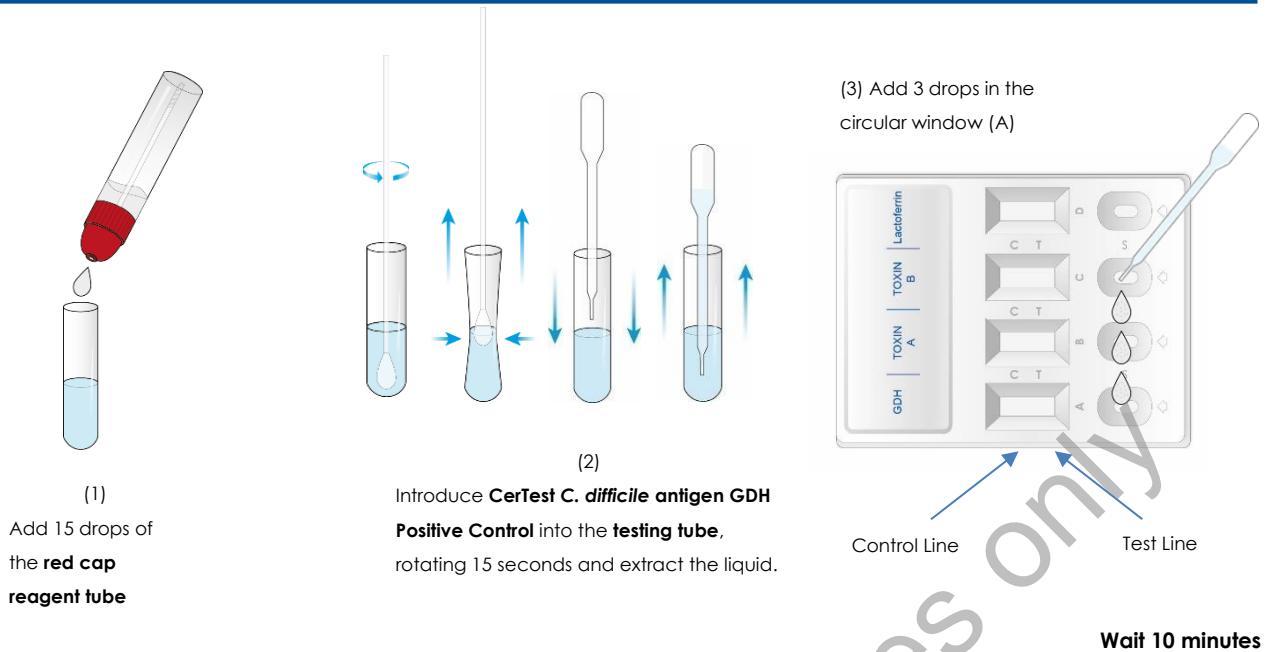
**CerTest C. difficile antigen GDH Positive Control** is an external quality control for **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (Strip A). Use **CerTest C. difficile antigen GDH Positive Control** to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C, **CerTest C. difficile antigen GDH Positive Control** is stable through the expiration date printed on the sealed pouch. **CerTest C. difficile antigen GDH Positive Control** must remain in the sealed pouch until use. Do not freeze.

##### 1.1. Positive control procedure

**Allow tests and CerTest C. difficile antigen GDH Positive Control to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.**

1. Take the **red cap reagent tube**, cut the end of the cap.
2. Add 15 drops (1) from the **red cap reagent tube** into the **testing tube** and immediately introduce **CerTest C. difficile antigen GDH Positive Control**.
3. Mix the solution by rotating **CerTest C. difficile antigen GDH Positive Control** forcefully against the side of the **testing tube** at least 15 seconds. Extract as much liquid as possible from the **CerTest C. difficile antigen GDH Positive Control**, squeezing the sides of the **testing tube** as **CerTest C. difficile antigen GDH Positive Control** is withdrawn (2). Discard **CerTest C. difficile antigen GDH Positive Control**.
4. Remove the **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (combo card format) from its sealed bag just before using it.
5. Use a separate **disposable pipette** and test for each control. Dispense 3 drops from the **testing tube** in the circular window marked with the letter A (3).
6. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

## 2. CerTest C. difficile Toxin A Positive Control

**CerTest C. difficile Toxin A Positive Control** is an external quality control for **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (Strip B). Use **CerTest C. difficile Toxin A Positive Control** to check that the extraction reagents and the test are working properly.

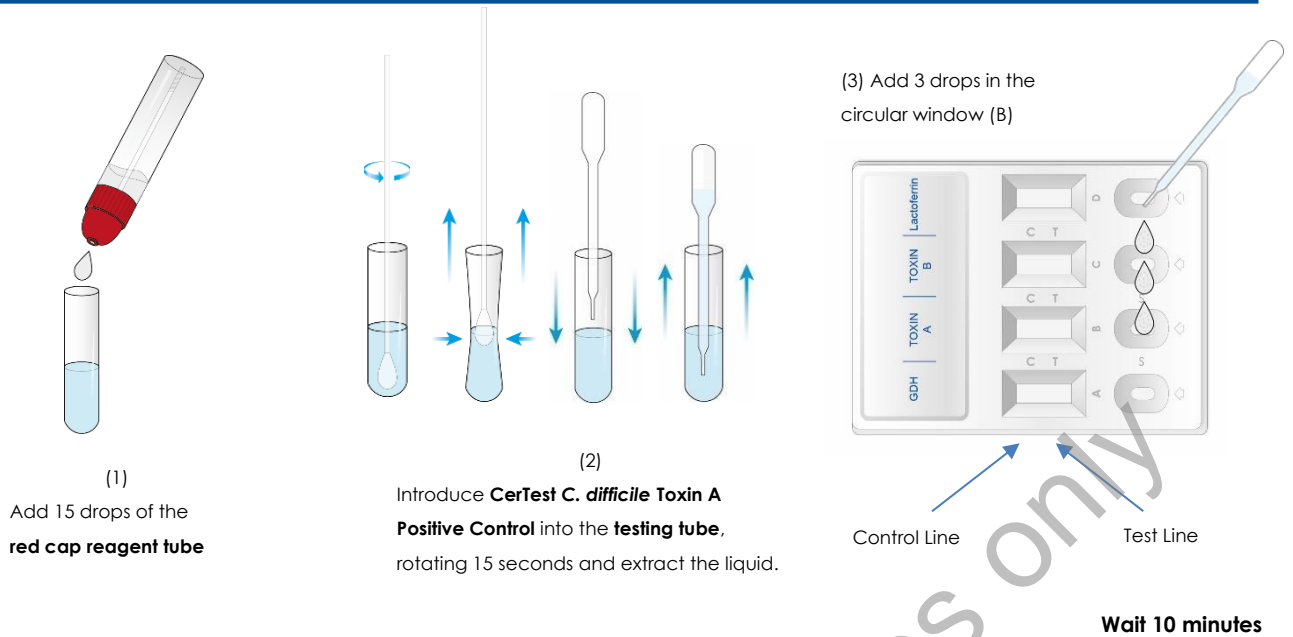
Store as packaged in the sealed pouch at 2-30°C, **CerTest C. difficile Toxin A Positive Control** is stable through the expiration date printed on the sealed pouch. **CerTest C. difficile Toxin A Positive Control** must remain in the sealed pouch until use. Do not freeze.

### 2.1. Positive control procedure

**Allow tests and CerTest C. difficile Toxin A Positive Control to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.**

1. Take the **red cap reagent tube**, cut the end of the cap.
2. Add 15 drops (1) from **red cap reagent tube** into the **testing tube** and immediately introduce **CerTest C. difficile Toxin A Positive Control**.
3. Mix the solution by rotating **CerTest C. difficile Toxin A Positive Control** forcefully against the side of the **testing tube** at least 15 seconds. Extract as much liquid as possible from **CerTest C. difficile Toxin A Positive Control**, squeezing the sides of the **testing tube** as **CerTest C. difficile Toxin A Positive Control** is withdrawn (2). Discard **CerTest C. difficile Toxin A Positive Control**.
4. Remove the **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (combo card format) from its sealed bag just before using it.
5. Use a separate **disposable pipette** and test for each control. Dispense 3 drops from the **testing tube** in the circular window marked with the letter B (3).
6. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

### 3. CerTest C. difficile Toxin B Positive Control

**CerTest C. difficile Toxin B Positive Control** is an external quality control for **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (Strip C). Use **CerTest C. difficile Toxin B Positive Control** to check that the extraction reagents and the test are working properly.

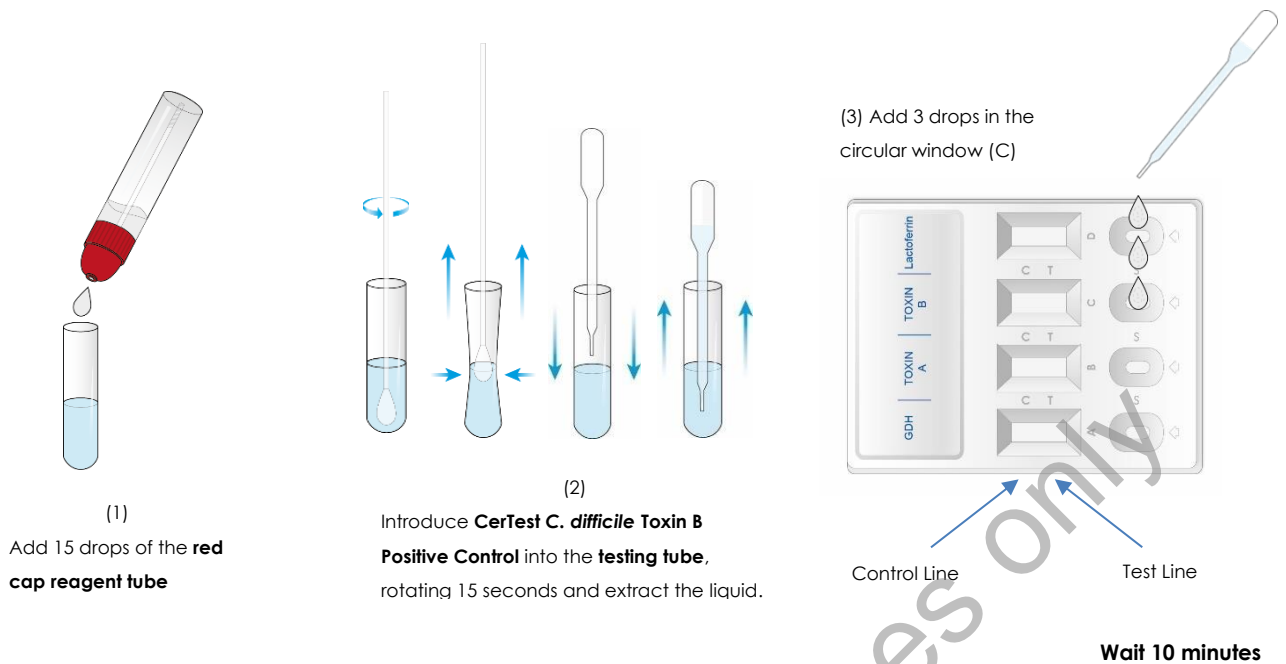
Store as packaged in the sealed pouch at 2-30°C, **CerTest C. difficile Toxin B Positive Control** is stable through the expiration date printed on the sealed pouch. **CerTest C. difficile Toxin B Positive Control** must remain in the sealed pouch until use. Do not freeze.

#### 4.1. Positive control procedure

Allow tests and **CerTest C. difficile Toxin B Positive Control** to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.

1. Take the **red cap reagent tube**, cut the end of the cap.
2. Add 15 drops (1) from the **red cap reagent tube** into the **testing tube** and immediately introduce the **CerTest C. difficile Toxin B Positive Control**.
3. Mix the solution by rotating the **CerTest C. difficile Toxin B Positive Control** forcefully against the side of the **Testing tube** at least 15 seconds. Extract as much liquid as possible from the **CerTest C. difficile Toxin B Positive Control**, squeezing the sides of the **testing tube** as the **CerTest C. difficile Toxin B Positive Control** is withdrawn (2). Discard **CerTest C. difficile Toxin B Positive Control**.
4. Remove the **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (combo card format) from its sealed bag just before using it.
5. Use a separate **disposable pipette** and test for each control. Dispense 3 drops from the **testing tube** in the circular window marked with the letter C (3).
6. **Read the results at 10 minutes**. Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

Positive controls could be tested once for each new test kit opened and as otherwise required by your laboratory's standard quality control procedures.

#### 4. CerTest Lactoferrin Positive Control

**CerTest Lactoferrin Positive Control** is an external quality control for **CerTest C. difficile GDH+Toxin A+B+Lactoferrin (Strip D)**. Use **CerTest Lactoferrin Positive Control** to check that the extraction reagents and the test are working properly.

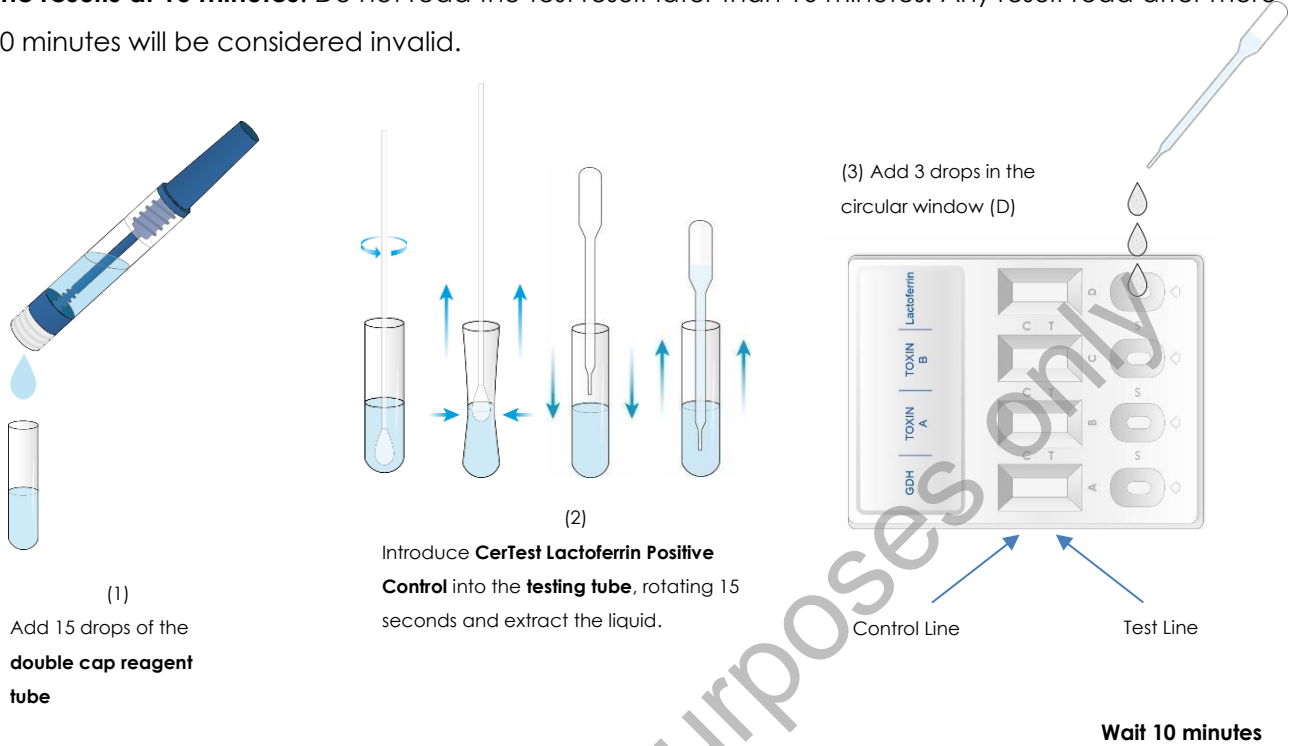
Store as packaged in the sealed pouch at 2-30°C, **CerTest Lactoferrin Positive Control** is stable through the expiration date printed on the sealed pouch. **CerTest Lactoferrin Positive Control** must remain in the sealed pouch until use. Do not freeze.

##### 4.1. Positive control procedure

**Allow tests and CerTest Lactoferrin Positive Control to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.**

1. Take the **double cap reagent tube**, open the cap 2.
2. Add 15 drops (1), from the **double cap reagent tube** into the **testing tube** and immediately introduce the **CerTest Lactoferrin Positive Control**.
3. Mix the solution by rotating the **CerTest Lactoferrin Positive Control** forcefully against the side of the **testing tube** at least 15 seconds. Extract as much liquid as possible from the **CerTest Lactoferrin Positive Control**, squeezing the sides of the **testing tube** as the **CerTest Lactoferrin Positive Control** is withdrawn (2). Discard **CerTest Lactoferrin Positive Control**.
4. Remove the **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (combo card format) from its sealed bag just before using it.

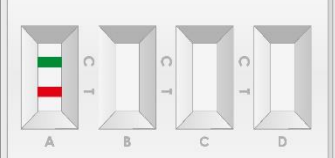
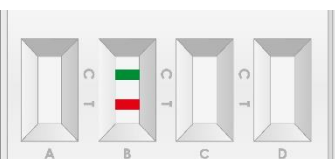
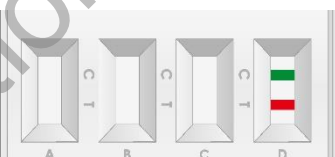
5. Use a separate **disposable pipette** and test for each control. Dispense 3 drops from the **testing tube** in the circular window marked with the letter D (3).
6. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

Positive controls could be tested once for each new test kit opened and as otherwise required by your laboratory's standard quality control procedures.

### A.1 Interpretation positive control procedure

	CerTest <i>C. difficile</i> antigen GDH Positive Control	CerTest <i>C. difficile</i> Toxin A Positive Control	CerTest <i>C. difficile</i> Toxin B Positive Control	CerTest Lactoferrin Positive Control	Interpretation of the results	
1.	<b>POSITIVE</b> (One device for four positive controls)				A GREEN line (control line (C)) and a RED line (test line (T)) in the four strips, in the results window, appear at the same time due to presence of antigens in <b>CerTest <i>C. difficile</i> antigen GDH Positive Control</b> , in strip A, in the <b>CerTest <i>C. difficile</i> Toxin A Positive Control</b> , in strip B, in <b>CerTest <i>C. difficile</i> Toxin B Positive Control</b> , in strip C, and in <b>CerTest Lactoferrin Positive Control</b> , in strip D.	
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN-RED		

	CerTest C. difficile antigen GDH Positive Control	CerTest C. difficile Toxin A Positive Control	CerTest C. difficile Toxin B Positive Control	CerTest Lactoferrin Positive Control		Interpretation of the results
2.	<b>POSITIVE</b> (One device for CerTest C. difficile antigen GDH Positive Control)					A GREEN line (control line (C)) and a RED line (test line (T)) appear across the results window in strip A due to presence of antigen GDH in <b>CerTest C. difficile antigen GDH Positive Control</b> .
	GREEN-RED	-	-	-		
3.	<b>POSITIVE</b> (One device for CerTest C. difficile Toxin A Positive Control)					A GREEN line (control line (C)) and a RED line (test line (T)) appear across the results window in strip B due to presence of Toxin A in <b>CerTest C. difficile Toxin A Positive Control</b> .
	-	GREEN-RED	-	-		
4.	<b>POSITIVE</b> (One device for CerTest C. difficile Toxin B Positive Control)					A GREEN line (control line (C)) and a RED line (test line (T)) appear across the results window in strip C due to presence of Toxin A in <b>CerTest C. difficile Toxin B Positive Control</b> .
	-	-	GREEN-RED	-		
5.	<b>POSITIVE</b> (One device for CerTest Lactoferrin Positive Control)					A GREEN line (control line (C)) and a RED line (test line (T)) appear across the results window in strip D due to presence of lactoferrin in <b>CerTest Lactoferrin Positive Control</b> .
	-	-	-	GREEN-RED		
6.	<b>INVALID</b>				<p>Invalid result either A, B, C or D, we recommend repeating the assay using the same sample with another test.</p>	<p>Total absence of the control-coloured line (GREEN), regardless the appearance or not of the test line (RED), or only the control (C) coloured line (GREEN). Incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the control with a new test and a new <b>CerTest C. difficile antigen GDH Positive Control</b>, <b>CerTest C. difficile Toxin A Positive Control</b>, <b>CerTest C. difficile Toxin B Positive Control</b> and <b>CerTest Lactoferrin Positive Control</b>.</p> <p>If the situation still persists, discontinue using the test kit and contact your local distributor.</p>
	Any other result					

CerTest C. <i>difficile</i> antigen GDH Positive Control	CerTest C. <i>difficile</i> Toxin A Positive Control	CerTest C. <i>difficile</i> Toxin B Positive Control	CerTest Lactoferrin Positive Control		Interpretation of the results
					Any result read after more than 10 minutes will be considered invalid.

Table A 2.3 CerTest C.*difficile* antigen GDH, Toxin A, Toxin B and Lactoferrin Positive Control performance e interpretation of the results.

For information purposes only

## ANNEX 3

**KIT 20 CARD C. DIFFICILE GDH+TOXIN A+B+LACTOFERRIN. Catalogue****reference: LX862001VFCC**

Annex for the following reference:

PRODUCT	REFERENCE
<b>CerTest C. difficile GDH+Toxin A+B+Lactoferrin</b> Combo Card (20 tests kit with stool vials +PC +NC)	LX862001VFCC

Table A 3. Reference for kit 20 combo card of CerTest C. *difficile* GDH+Toxin A+B+Lactoferrin with positive and negative control.**I. Materials**

MATERIALS PROVIDED	MATERIALS REQUIRED BUT NO PROVIDED
<ul style="list-style-type: none"> <li>- 20x <b>CerTest C. difficile GDH+Toxin A+B+Lactoferrin</b> (combo card test format)</li> <li>- 20x <b>GDH+Toxin A+B stool collection tubes</b> with diluent</li> <li>- 20x <b>Lactoferrin stool collection tubes</b> with diluent</li> <li>- 1x <b>CerTest C. difficile antigen GDH Positive Control</b></li> <li>- 1x <b>CerTest C. difficile Toxin A Positive Control</b></li> <li>- 1x <b>CerTest C. difficile Toxin B Positive Control</b></li> <li>- 1x <b>CerTest Lactoferrin Positive Control</b></li> <li>- 1x <b>CerTest Negative Control</b></li> <li>- 6x <b>Testing tube</b></li> <li>- 6x <b>Disposable pipette</b></li> <li>- 4x <b>Red cap reagent tube</b></li> <li>- 2x <b>Double cap reagent tube</b></li> <li>- 1x IFU Instructions for use (including Quick guide use).</li> </ul>	<ul style="list-style-type: none"> <li>- Disposable gloves (other personal protective equipment that will be considered necessary)</li> <li>- Timer</li> <li>- Specimen collection container</li> </ul>


Table A 3.1 Materials necessities for test performance.

## II. Test procedures (samples)


Follow proper infection control practices. Allow tests and samples to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Proceed to shake the **stool collection tube** (both types) in order to assure good sample dispersion.
2. Remove **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (combo card test) from its sealed bag just before using it.
3. Take the **GDH+Toxin A+B stool collection tube**, cut the end of the cap (1) and dispense 3 drops in the circular window marked with the letter A (2), 3 drops, using the same tube, in the circular window marked with letter B (3) and 3 drops, using the same tube, in the circular window marked with letter C (4). Avoid adding solid particles with the liquid.
4. Take the **Lactoferrin stool collection tube**, open the cap 2 (5) and dispense 3 drops in the circular window marked with the letter D (6). Avoid adding solid particles with the liquid.
5. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

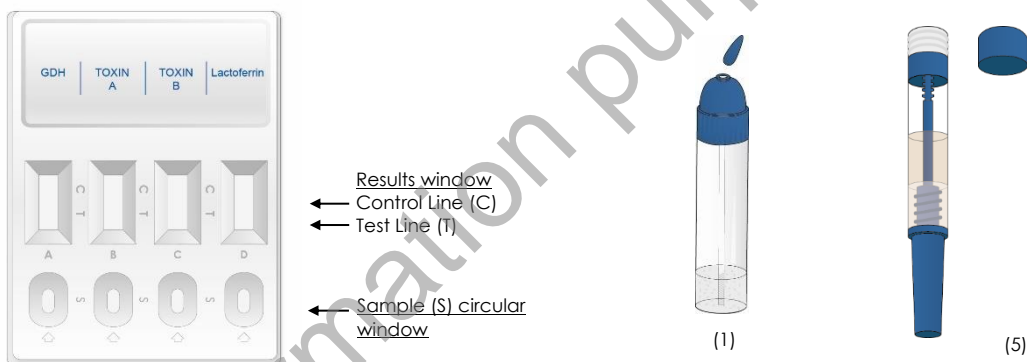
If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of diluent until seeing the liquid running through the reaction zone.

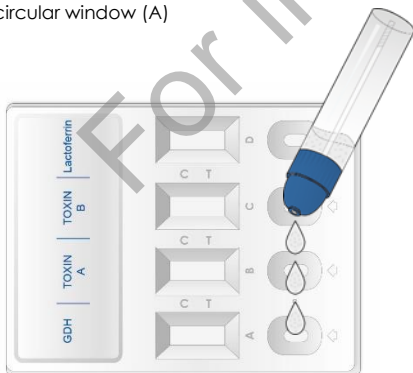
(2) Add 3 drops in the circular window (A)

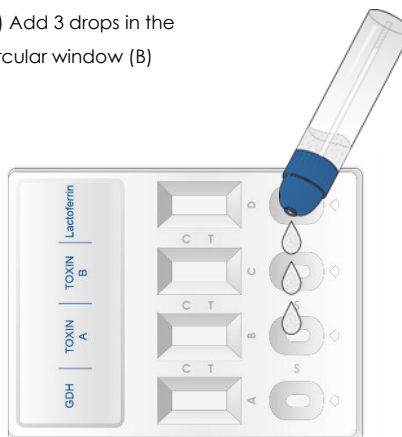
(3) Add 3 drops in the circular window (B)

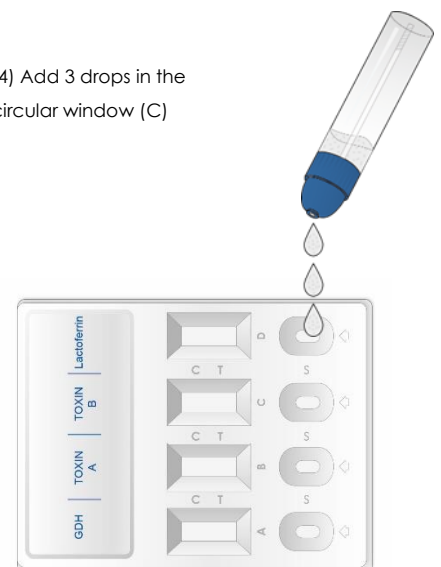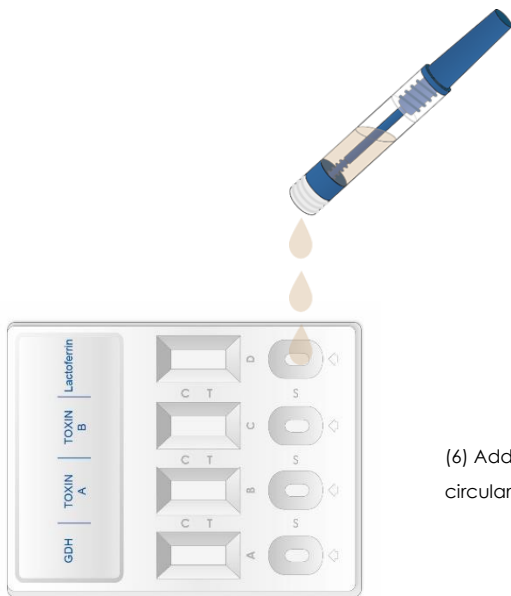(4) Add 3 drops in the circular window (C)






(6) Add 3 drops in the circular window (D)

Wait 10 minutes






### III. Interpretation of results

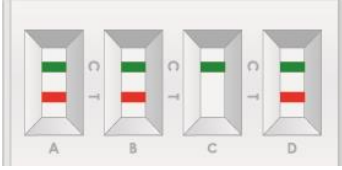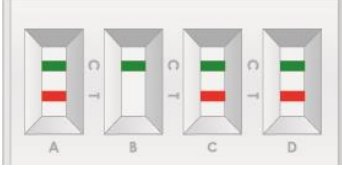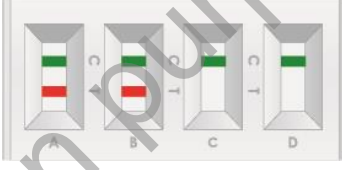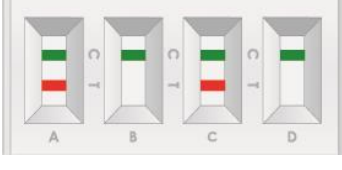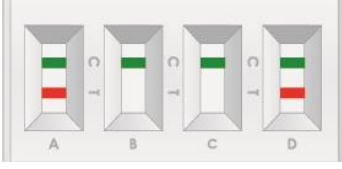(please, refer to the illustration below)

	GDH	Toxin A	Toxin B	Lactoferrin		Interpretation of the results
1.	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE		<p>There are no GDH, Toxin A or Toxin B of <i>C. difficile</i> presence and neither human lactoferrin. No infection caused by <i>Clostridium difficile</i>.</p> <p>A negative result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN	GREEN	GREEN	GREEN		
2.	POSITIVE	POSITIVE	POSITIVE	POSITIVE		<p>There are GDH, Toxin A, Toxin B and Lactoferrin presence. Possible infection caused by <i>Clostridium difficile</i> and there is an intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN-RED		
3.	POSITIVE	POSITIVE	POSITIVE	NEGATIVE		<p>There is GDH, Toxin A and Toxin B presence. Possible infection caused by <i>Clostridium difficile</i>.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN		
	POSITIVE	POSITIVE	NEGATIVE	POSITIVE		

	GDH	Toxin A	Toxin B	Lactoferrin		Interpretation of the results
4.	GREEN-RED	GREEN-RED	GREEN	GREEN-RED		<p>There is GDH, Toxin A and Lactoferrin presence. Possible infection caused by <i>Clostridium difficile</i>, and there is an intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
5.	POSITIVE	NEGATIVE	POSITIVE	POSITIVE		<p>There is GDH, Toxin B and Lactoferrin presence. Possible infection caused by <i>Clostridium difficile</i>, and there is an intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN	GREEN-RED	GREEN-RED		
6.	POSITIVE	POSITIVE	NEGATIVE	NEGATIVE		<p>There is GDH and Toxin A presence. Possible infection caused by <i>Clostridium difficile</i>.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN-RED	GREEN	GREEN		
7.	POSITIVE	NEGATIVE	POSITIVE	NEGATIVE		<p>There is GDH and Toxin B presence. Possible infection caused by <i>Clostridium difficile</i>.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN	GREEN-RED	GREEN		
8.	POSITIVE	NEGATIVE	NEGATIVE	POSITIVE		<p>There is GDH and Lactoferrin presence. Possible infection caused by <i>Clostridium difficile</i> and there is an intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN-RED	GREEN	GREEN	GREEN-RED		
9.	POSITIVE	NEGATIVE	NEGATIVE	NEGATIVE		

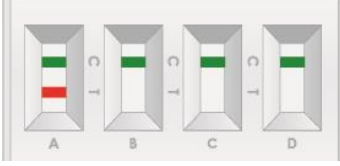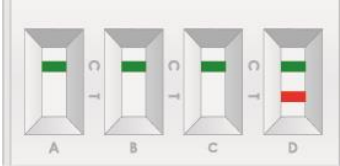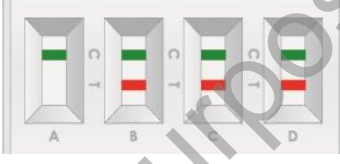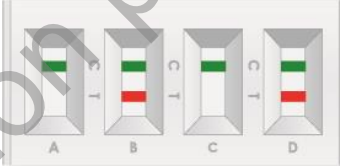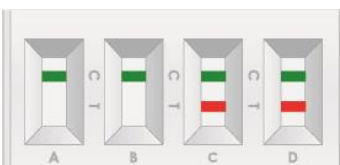	GDH	Toxin A	Toxin B	Lactoferrin		Interpretation of the results
	GREEN-RED	GREEN	GREEN	GREEN		<p>There is GDH presence. Possible infection caused by <i>Clostridium difficile</i>.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
10.	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE		<p>There is Lactoferrin presence. Possible intestinal inflammation.</p> <p>The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.</p> <p>A positive result should be followed up with additional laboratory techniques to confirm the results.</p>
	GREEN	GREEN	GREEN	GREEN-RED		<p>There is Toxin A, Toxin B and Lactoferrin presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
11.	NEGATIVE	POSITIVE	POSITIVE	POSITIVE		<p>There is Toxin A and Lactoferrin presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN	GREEN	GREEN-RED		<p>There is Toxin A and Lactoferrin presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
12.	NEGATIVE	POSITIVE	NEGATIVE	POSITIVE		<p>There is Toxin A and Lactoferrin presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN-RED	GREEN	GREEN-RED		<p>There is Toxin B and Lactoferrin presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
13.	NEGATIVE	NEGATIVE	POSITIVE	POSITIVE		<p>There is Toxin A and Toxin B presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN	GREEN-RED	GREEN-RED		<p>There is Toxin A and Toxin B presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
14.	NEGATIVE	POSITIVE	POSITIVE	NEGATIVE		<p>There is Toxin A presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN-RED	GREEN-RED	GREEN		<p>There is Toxin A presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
15.	NEGATIVE	POSITIVE	NEGATIVE	NEGATIVE		<p>There is Toxin A presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN-RED	GREEN	GREEN		<p>There is Toxin A presence.</p> <p>If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>

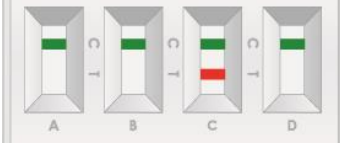
	GDH	Toxin A	Toxin B	Lactoferrin		Interpretation of the results
16.	NEGATIVE	NEGATIVE	POSITIVE	NEGATIVE		<p>There is Toxin B presence. If this result appears it must be repeat the test using a fresh sample. If results are again positive for Toxin A and B and negative for GDH, the sample should be considered positive for Toxin A and B.</p>
	GREEN	GREEN	GREEN-RED	GREEN		
17.	INVALID				<p>Invalid result either A, B, C or D, we recommend repeating the assay using the same sample with another test.</p> <p>Any result read after more than 10 minutes will be considered invalid.</p>	
	Any other results					

Table A 3.2 CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin performance interpretation of the results.

Whatever results detailed between 11 and 16, are not very common. A very low percentage of specimens might result negative for GDH but positive for toxins.

**NEGATIVE:** Only one GREEN line appears across the results window in the site marked with the letter C (control line).

**POSITIVE:** In addition to the GREEN control line, a RED line also appears in the site marked with the letter T (test line) in the results window.

The result would be considered as positive independent on the intensity of the RED test line.

**INVALID:** Total absence of the control coloured line (GREEN) regardless the appearance or not of the test line (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persists, discontinue using the test kit and contact your local distributor.

#### NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red coloured band in the test line (T) in the results window will vary depending on the concentration of antigen presents in the specimen. However, neither the quantitative value nor the rate of increase in antigen can be determined by this qualitative test.

## IV. Quality control

**CerTest *C. difficile* GDH+Toxin A+B+Lactoferrin** (combo card test) presents six controls:

INTERNAL CONTROL: control line

**Internal control:** internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

**EXTERNAL CONTROL: POSITIVE CONTROL SWABS AND NEGATIVE CONTROL SWAB**

**External control: CerTest C. difficile antigen GDH Positive Control, CerTest C. difficile Toxin A Positive Control, CerTest C. difficile Toxin B Positive Control and CerTest Lactoferrin Positive Control** (positive control swabs).

**External negative control: CerTest Negative Control** (negative control swab)

**A. External positive controls**

It is recommended to use the same device for both external positive controls.

**1. CerTest C. difficile antigen GDH Positive Control**

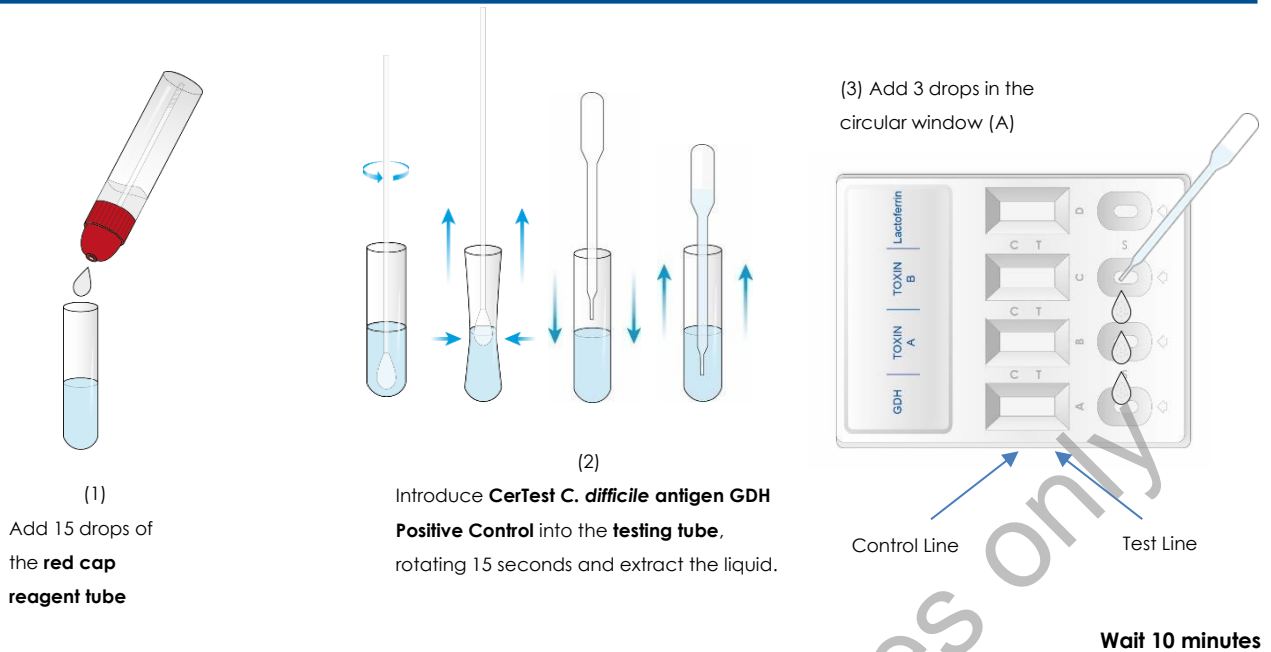
**CerTest C. difficile antigen GDH Positive Control** is an external quality control for **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (Strip A). Use **CerTest C. difficile antigen GDH Positive Control** to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C, **CerTest C. difficile antigen GDH Positive Control** is stable through the expiration date printed on the sealed pouch. **CerTest C. difficile antigen GDH Positive Control** must remain in the sealed pouch until use. Do not freeze.

**1.1. Positive control procedure**

**Allow tests and CerTest C. difficile antigen GDH Positive Control to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.**

1. Take the **red cap reagent tube**, cut the end of the cap.
2. Add 15 drops (1) from the **red cap reagent tube** into the **testing tube** and immediately introduce **CerTest C. difficile antigen GDH Positive Control**.
3. Mix the solution by rotating **CerTest C. difficile antigen GDH Positive Control** forcefully against the side of the **testing tube** at least 15 seconds. Extract as much liquid as possible from the **CerTest C. difficile antigen GDH Positive Control**, squeezing the sides of the **testing tube** as **CerTest C. difficile antigen GDH Positive Control** is withdrawn (2). Discard **CerTest C. difficile antigen GDH Positive Control**.
4. Remove the **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (combo card format) from its sealed bag just before using it.
5. Use a separate **disposable pipette** and test for each control. Dispense 3 drops from the **testing tube** in the circular window marked with the letter A (3).
6. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

## 2. CerTest C. difficile Toxin A Positive Control

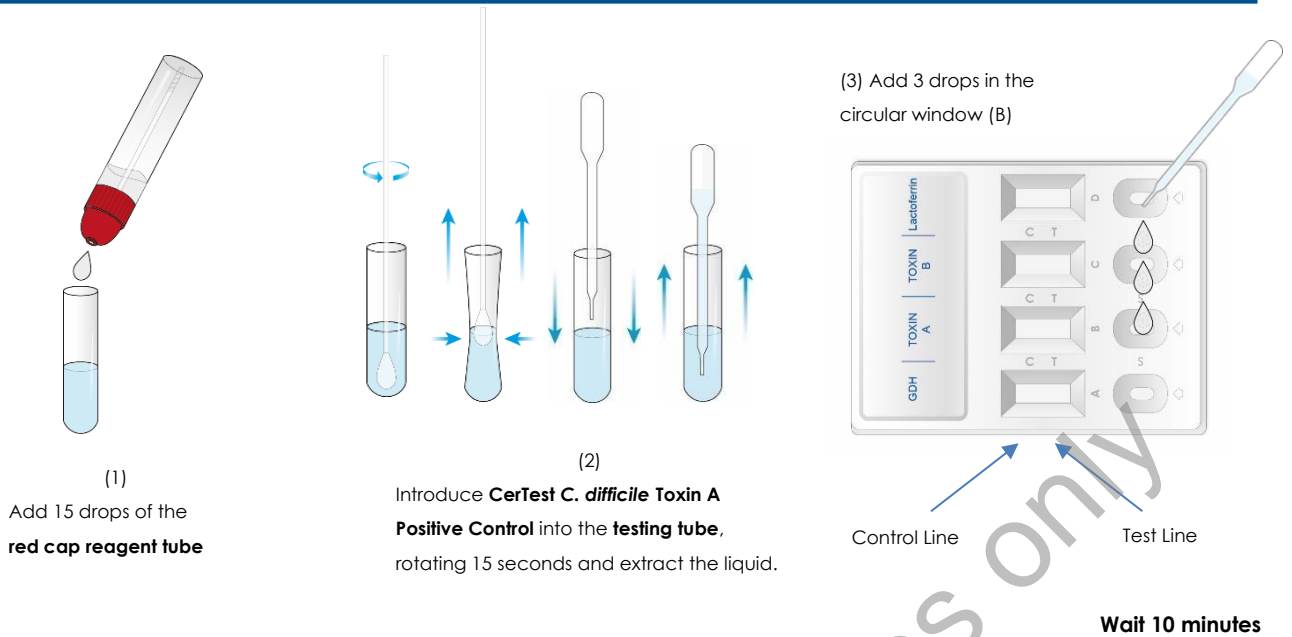
**CerTest C. difficile Toxin A Positive Control** is an external quality control for **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (Strip B). Use **CerTest C. difficile Toxin A Positive Control** to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C, **CerTest C. difficile Toxin A Positive Control** is stable through the expiration date printed on the sealed pouch. **CerTest C. difficile Toxin A Positive Control** must remain in the sealed pouch until use. Do not freeze.

### 2.1. Positive control procedure

**Allow tests and CerTest C. difficile Toxin A Positive Control to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.**

1. Take the **red cap reagent tube**, cut the end of the cap.
2. Add 15 drops (1) from **red cap reagent tube** into the **testing tube** and immediately introduce **CerTest C. difficile Toxin A Positive Control**.
3. Mix the solution by rotating **CerTest C. difficile Toxin A Positive Control** forcefully against the side of the **testing tube** at least 15 seconds. Extract as much liquid as possible from **CerTest C. difficile Toxin A Positive Control**, squeezing the sides of the **testing tube** as **CerTest C. difficile Toxin A Positive Control** is withdrawn (2). Discard **CerTest C. difficile Toxin A Positive Control**.
4. Remove the **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (combo card format) from its sealed bag just before using it.
5. Use a separate **disposable pipette** and test for each control. Dispense 3 drops from the **testing tube** in the circular window marked with the letter B (3).
6. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

### 3. CerTest C. difficile Toxin B Positive Control

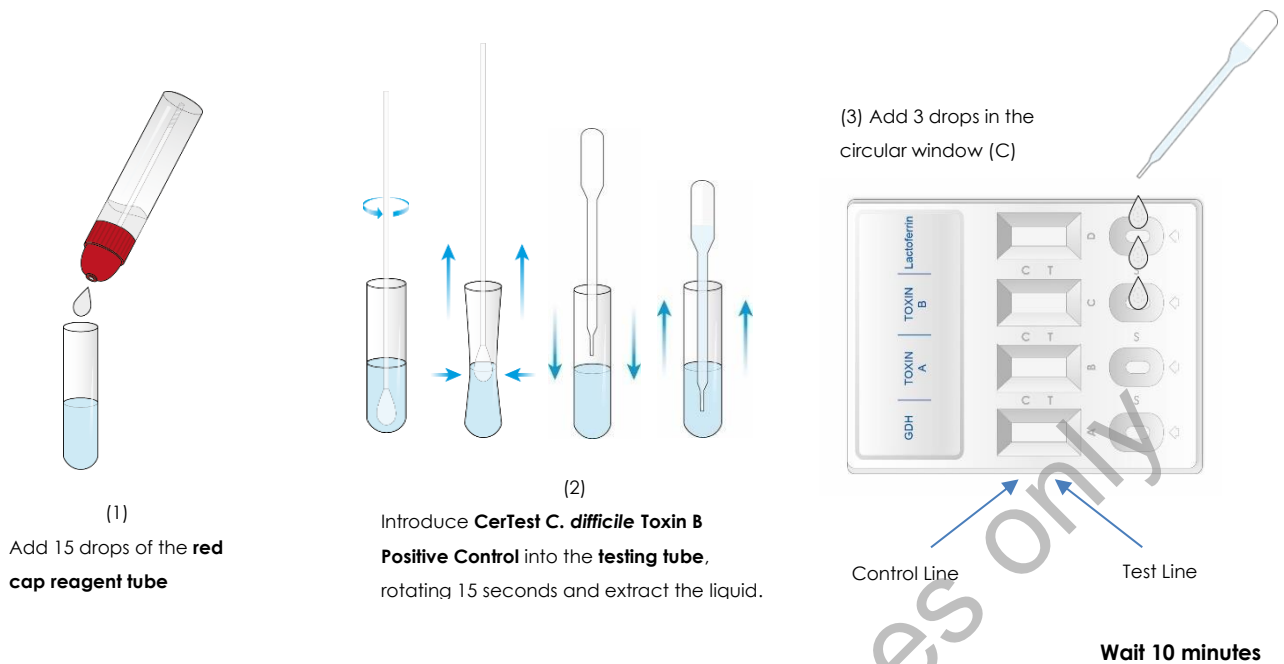
**CerTest C. difficile Toxin B Positive Control** is an external quality control for **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (Strip C). Use **CerTest C. difficile Toxin B Positive Control** to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C, **CerTest C. difficile Toxin B Positive Control** is stable through the expiration date printed on the sealed pouch. **CerTest C. difficile Toxin B Positive Control** must remain in the sealed pouch until use. Do not freeze.

#### 4.1. Positive control procedure

Allow tests and **CerTest C. difficile Toxin B Positive Control** to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.

1. Take the **red cap reagent tube**, cut the end of the cap.
2. Add 15 drops (1) from the **red cap reagent tube** into the **testing tube** and immediately introduce the **CerTest C. difficile Toxin B Positive Control**.
3. Mix the solution by rotating the **CerTest C. difficile Toxin B Positive Control** forcefully against the side of the **testing tube** at least 15 seconds. Extract as much liquid as possible from the **CerTest C. difficile Toxin B Positive Control**, squeezing the sides of the **testing tube** as the **CerTest C. difficile Toxin B Positive Control** is withdrawn (2). Discard **CerTest C. difficile Toxin B Positive Control**.
4. Remove the **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (combo card format) from its sealed bag just before using it.
5. Use a separate **disposable pipette** and test for each control. Dispense 3 drops from the **testing tube** in the circular window marked with the letter C (3).
6. **Read the results at 10 minutes**. Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

Positive controls could be tested once for each new test kit opened and as otherwise required by your laboratory's standard quality control procedures.

#### 4. CerTest Lactoferrin Positive Control

**CerTest Lactoferrin Positive Control** is an external quality control for **CerTest C. difficile GDH+Toxin A+B+Lactoferrin (Strip D)**. Use **CerTest Lactoferrin Positive Control** to check that the extraction reagents and the test are working properly.

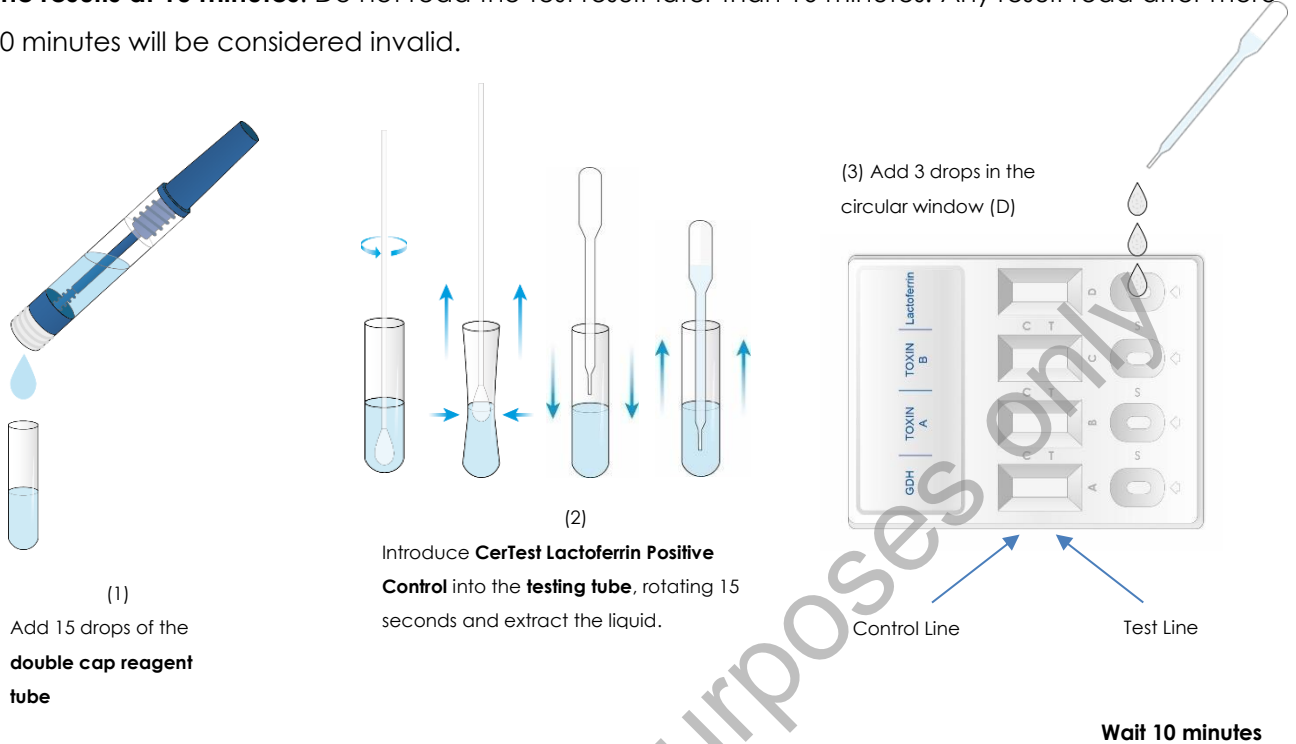
Store as packaged in the sealed pouch at 2-30°C, **CerTest Lactoferrin Positive Control** is stable through the expiration date printed on the sealed pouch. **CerTest Lactoferrin Positive Control** must remain in the sealed pouch until use. Do not freeze.

##### 4.1. Positive control procedure

**Allow tests and CerTest Lactoferrin Positive Control to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.**

1. Take the **double cap reagent tube**, open the cap 2.
2. Add 15 drops (1), from the **double cap reagent tube** into the **testing tube** and immediately introduce the **CerTest Lactoferrin Positive Control**.
3. Mix the solution by rotating the **CerTest Lactoferrin Positive Control** forcefully against the side of the **Testing tube** at least 15 seconds. Extract as much liquid as possible from the **CerTest Lactoferrin Positive Control**, squeezing the sides of the **testing tube** as the **CerTest Lactoferrin Positive Control** is withdrawn (2). Discard **CerTest Lactoferrin Positive Control**.
4. Remove the **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (combo card format) from its sealed bag just before using it.

5. Use a separate **disposable pipette** and test for each control. Dispense 3 drops from the **testing tube** in the circular window marked with the letter D (3).
6. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

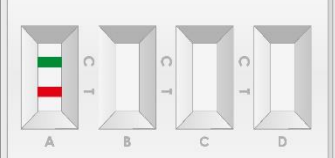
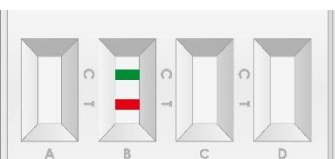
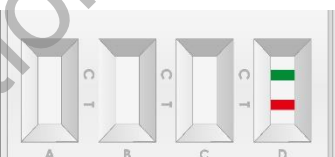
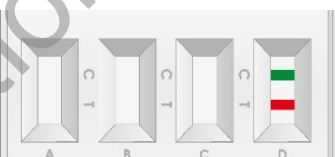
Positive controls could be tested once for each new test kit opened and as otherwise required by your laboratory's standard quality control procedures.

### A.1 Interpretation positive control procedure

	CerTest C. difficile antigen GDH Positive Control	CerTest C. difficile Toxin A Positive Control	CerTest C. difficile Toxin B Positive Control	CerTest Lactoferrin Positive Control	Interpretation of the results	
1.	<b>POSITIVE</b> (One device for four positive controls)					A GREEN line (control line (C)) and a RED line (test line (T)) in the four strips, in the results window, appear at the same time due to presence of antigens in <b>CerTest C. difficile antigen GDH Positive Control</b> , in strip A, in the <b>CerTest C. difficile Toxin A Positive Control</b> , in strip B, in <b>CerTest C. difficile Toxin B Positive Control</b> , in strip C, and in <b>CerTest Lactoferrin Positive Control</b> , in strip D.
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN-RED		

	CerTest C. difficile antigen GDH Positive Control	CerTest C. difficile Toxin A Positive Control	CerTest C. difficile Toxin B Positive Control	CerTest Lactoferrin Positive Control		Interpretation of the results
2.	<b>POSITIVE</b> (One device for CerTest C. difficile antigen GDH Positive Control)					A GREEN line (control line (C)) and a RED line (test line (T)) appear across the results window in strip A due to presence of antigen GDH in <b>CerTest C. difficile antigen GDH Positive Control</b> .
	GREEN-RED	-	-	-		
3.	<b>POSITIVE</b> (One device for CerTest C. difficile Toxin A Positive Control)					A GREEN line (control line (C)) and a RED line (test line (T)) appear across the results window in strip B due to presence of Toxin A in <b>CerTest C. difficile Toxin A Positive Control</b> .
	-	GREEN-RED	-	-		
4.	<b>POSITIVE</b> (One device for CerTest C. difficile Toxin B Positive Control)					A GREEN line (control line (C)) and a RED line (test line (T)) appear across the results window in strip C due to presence of Toxin B in <b>CerTest C. difficile Toxin B Positive Control</b> .
	-	-	GREEN-RED	-		
5.	<b>POSITIVE</b> (One device for CerTest Lactoferrin Positive Control)					A GREEN line (control line (C)) and a RED line (test line (T)) appear across the results window in strip D due to presence of lactoferrin in <b>CerTest Lactoferrin Positive Control</b> .
	-	-	-	GREEN-RED		
6.	<b>INVALID</b>					Total absence of the control-coloured line (GREEN), regardless the appearance or not of the test line (RED), or only the control (C) coloured line (GREEN). Incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the control with a new test and a new <b>CerTest C. difficile antigen GDH Positive Control, CerTest C. difficile Toxin A Positive Control, CerTest C. difficile Toxin B Positive Control and CerTest Lactoferrin Positive Control</b> .
	Any other result					

CerTest C. <i>difficile</i> antigen GDH Positive Control	CerTest C. <i>difficile</i> Toxin A Positive Control	CerTest C. <i>difficile</i> Toxin B Positive Control	CerTest Lactoferrin Positive Control	Interpretation of the results
				Any result read after more than 10 minutes will be considered invalid.

Table A 2.3 CerTest C.*difficile* antigen GDH, Toxin A, Toxin B and Lactoferrin Positive Control performance e interpretation of the results.

## B. External negative control. CerTest Negative Control

**CerTest Negative Control** is an external quality control for **CerTest C. *difficile* GDH+Toxin A+B+Lactoferrin** (Strip A, B, C and D). Use **CerTest Negative Control** to check that the extraction reagents and the test are working properly.

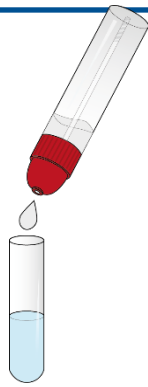
Store as packaged in the sealed pouch at 2-30°C, **CerTest Negative Control** is stable through the expiration date printed on the sealed pouch. **CerTest Negative Control** must remain in the sealed pouch until use. Do not freeze.

### B.1 Negative control procedure

**Allow tests and CerTest Negative Control to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.**


**For GDH, Toxin A and Toxin B:**

1. Add 15 drops (1) of diluent from the **red cap reagent tube** into the **testing tube** and immediately put **CerTest Negative Control**.
2. Mix the solution by rotating **CerTest Negative Control** forcefully against the side of the **testing tube** at least 15 seconds. Extract as much liquid as possible from the **CerTest Negative Control**, squeezing the sides of the **testing tube** as **CerTest Negative Control** is withdrawn (2). Discard the **CerTest Negative Control**.
3. Remove the **CerTest C. *difficile* GDH+Toxin A+B+Lactoferrin** (combo card format) from its sealed bag just before using it.
4. Use a separate **disposable pipette** and test for each control. Dispense 3 drops from the **testing tube** into the circular window marked with letter A (3) dispense 3 drops from the **testing tube** into the circular window marked with letter B (4), and dispense 3 drops from the **testing tube** into the circular window marked with letter C (5).
5. **Read the result at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

(1)

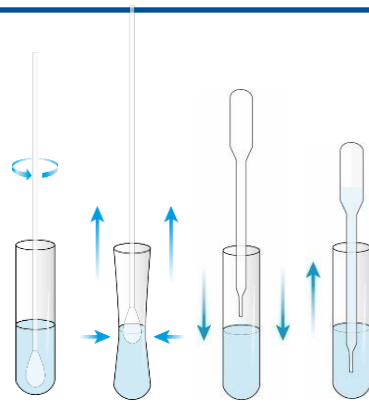
Add 15 drops of **red cap reagent tube**

(2)

Introduce the **CerTest Negative Control** into the **testing tube**, rotating 15 seconds and extract the liquid

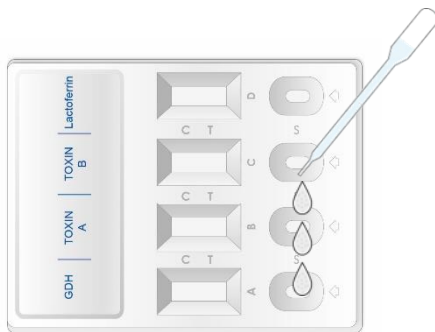
(3) Add 3 drops in the circular window (A)

Control Line

Test Line

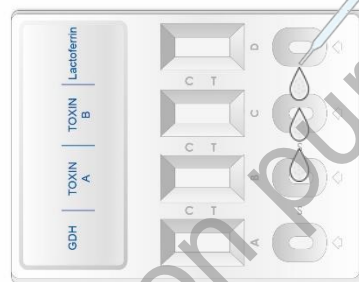
(4) Add 3 drops in the circular window (B)

Control Line

Test Line

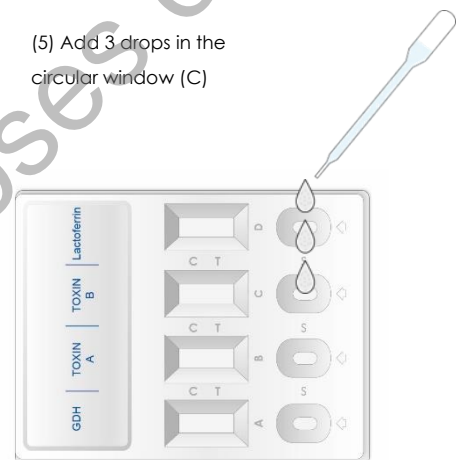(5) Add 3 drops in the circular window (C)

Control Line

Test Line

Wait 10 minutes

### For Lactoferrin:

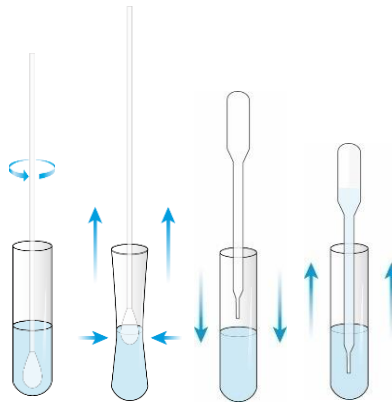
1. Add 15 drops (1) of diluent from the **double cap reagent tube** into the **testing tube** and immediately put **CerTest Negative Control**.
2. Mix the solution by rotating **CerTest Negative Control** forcefully against the side of the **testing tube** at least 15 seconds. Extract as much liquid as possible from the **CerTest Negative Control**, squeezing the sides of the **testing tube** as **CerTest Negative Control** is withdrawn (2). Discard the **CerTest Negative Control**.
3. Remove the **CerTest C. difficile GDH+Toxin A+B+Lactoferrin** (combo card format) from its sealed bag just before using it.
4. Use a separate **disposable pipette** and test for each control. Dispense 3 drops from the **testing tube** into the circular window marked with letter D (3).

5. **Read the result at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

(1)


Add 15 drops of **double cap reagent tube**

(2)

Introduce the **CerTest Negative Control** into the **testing tube**, rotating 15 seconds and extract the liquid

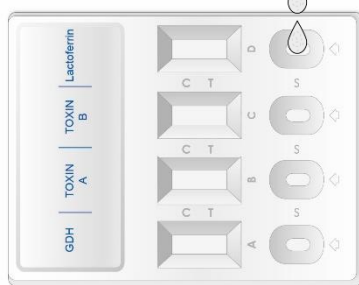(3) Add 3 drops in the circular window (D)

Control Line

Test Line

**Wait 10 minutes**

Negative controls could be tested once for each new test kit opened and as otherwise required by your laboratory's standard quality control procedures.

**B.1.1 Interpretation negative control procedure**

Negative Control			Interpretation of the results
1.	<b>VALID RESULT</b>		Only A GREEN line (control line (C)) appear across the results window during the test performance.
	GREEN (only control band appear)		
2.	<b>INVALID</b>	Invalid result either A, B, C or D, we recommend repeating the assay using the same sample with another test.	<p>Total absence of the control-coloured line (GREEN), regardless the appearance or not of the test line (RED), or present of both lines (RED test and GREEN control lines). Incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the control with a new test device and a new <b>CerTest Negative Control</b>.</p> <p>If the situation still persists, discontinue using the test kit and contact your local distributor.</p>
	Any other result		

Table A 3.4 CerTest Negative Control performance interpretation of the results.

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## 11. Reference/Bibliografía

1. Vaishnavi C. **Clinical spectrum & pathogenesis of Clostridium difficile associated diseases.** Indian J Med Res. 2010;131(4):487–99.
2. Fitzsimmons. **Independent evaluation of a GDH enzyme immunoassay for C. difficile screening.** Biomed Sci. 2011;(October):698–700.
3. Ofori E, Ramai D, Dhawan M, Mustafa F, Gasperino J, Reddy M. **Community-acquired Clostridium difficile: epidemiology, ribotype, risk factors, hospital and intensive care unit outcomes, and current and emerging therapies.** J Hosp Infect. 2018;99(4):436–42.
4. Guh AY, Mu Y, Winston LG, Johnston H, Olson D, Farley MM, et al. **Trends in U.S. Burden of Clostridioides difficile Infection and Outcomes.** N Engl J Med. 2021;382(14):1320–30.
5. Wren MWD, Sivapalan M, Kinson R, Shetty NP. **An evaluation of tests for faecal toxin , glutamate dehydrogenase , lactoferrin and toxigenic culture in the diagnostic laboratory.** Br J Biomed Sci. 2009;66(1):1–5.
6. Angriman I, Scarpa M, D'Incà R, Basso D, Ruffolo C, Polese L, et al. **Enzymes in feces: Useful markers of chronic inflammatory bowel disease.** Clin Chim Acta. 2007;381(1):63–8.
7. Guerrant RL, Araujo V, Soares E, Kotloff K, Lima AAM, Cooper WH, et al. **Measurement of fecal lactoferrin as a marker of fecal leukocytes.** J Clin Microbiol. 1992;30(5):1238–42.
8. Services U. D of H and H. **Antibiotic resistance threats in the United States.** Centers Dis Control Prev. 2019;1–113.

## 12. Symbols for IVD components and reagents/Símbolos para reactivos y productos para diagnóstico in vitro

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